



Pairing and connecting iCom with your Nokia mobile phone

**1. Charge and turn on both mobile phone and iCom.
Place them next to each other.**

2. Prepare your mobile phone for pairing

Find the connectivity settings of your phone. With most Nokia models the following navigation applies.

Menu ▷ Settings ▷ Connectivity ▷ Bluetooth

3. Prepare your iCom for pairing

Initiate the pairing process on your iCom by pressing the Bluetooth pairing button for about 2 seconds until the Bluetooth indicator flashes blue. Your iCom will remain in pairing mode for two minutes or until the pairing is completed.

4. Pair iCom and mobile phone

On your mobile phone, still in the Bluetooth menu, select one of the following (depending on the model):

Search for audio enhancement

or

Press right to switch to the Paired devices tab ▷
Options ▷ New paired device

(If you have not activated the Bluetooth functionality, you may be asked if you wish to turn on Bluetooth. Doing so will start the search process for Bluetooth devices.)

Your mobile phone should present a list of Bluetooth devices discovered.

Select "iCom" from this list.

Enter passcode "0000" (four zeros).

iCom will be added to the Paired devices list.

5. Connect iCom with mobile phone

Still on your mobile phone, in the Paired devices menu, select iCom and select

Connect

or

Connect to audio device

Some phone models will present a dialog similar to the following: "Allow this device to connect automatically? Choosing "Yes" allows connections to take place automatically without confirmation. Choosing "No" will require your confirmation."

We recommend that you enable this feature if available.

If you have difficulties navigating to the Bluetooth settings, please refer to the mobile phone's user manual or ask the mobile phone dealer.