


# iCom Bluetooth Pairing Guides Mobile Phones

PHONAK

life is on

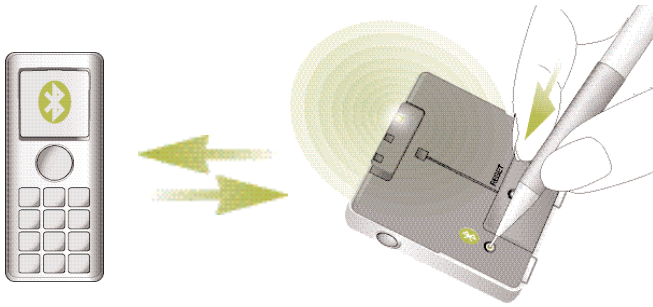

## Pairing and connecting iCom with your mobile phone

Charge and turn on both mobile phone and iCom.  
Place them next to each other.

### 1. Turn on Bluetooth

Find the connectivity settings of the phone. Look for the "Bluetooth" feature in the menu or one of its submenus and turn the Bluetooth feature on.

If the Bluetooth settings aren't in the menu, they may be found under "Options", "Tools", "Settings", "Connections", "Connectivity", "Applications", "Control Panel", "Setup", or under a combination of any of these (e.g. "Menu" ▷ "Settings" ▷ "Connectivity" ▷ "Bluetooth").

### 2. Set iCom in pairing mode

To initiate the pairing process on iCom, set iCom into pairing mode by pressing and holding the Bluetooth button for 2 seconds until the Bluetooth indicator LED is blue and flashing fast. iCom is now in pairing mode for 2 minutes or until the pairing is completed.


### 3. Search for iCom and confirm pairing with passcode 0000 (four zeros)

On your mobile phone, choose to add a new Bluetooth device. The mobile phone will start searching for Bluetooth enabled devices in range.

In the list of available devices, select "iCom" and enter "0000" (four zeros) when asked for the passcode or passkey. Allow devices to complete the pairing process.

On the mobile phone, iCom will be shown as a paired device and on iCom the Bluetooth indicator LED will flash slowly, indicating that the devices have been successfully paired.

### 4. Connect

In addition to being paired, which is done only once, your iCom and mobile phone need to be connected. This is done either automatically or via the mobile phone's menu. In the Bluetooth feature menu, select "Paired devices", or "My devices", highlight "iCom" and select "Connect". Some mobile phones may ask which Bluetooth service you wish to enable. Select "headset" and if available "stereo headphones". On some mobile phones, the headset icon  indicates that iCom is connected.

If prompted to allow automatic reconnection, we recommend to do so. This way iCom and your mobile phone will (re)connect whenever in range.

#### Mobile phone compatibility issues

The Bluetooth functionality is implemented in different ways by the various mobile phone manufacturers.

Unfortunately, in some mobile phone models, even with Bluetooth functionality, they may be compatible only with certain types of devices. If a problem occurs in the connection between your mobile phone and iCom, or if you are uncertain regarding your mobile phone's Bluetooth capabilities, please consult your mobile phone manual or ask your mobile phone dealer.