


iCom Bluetooth Pairing Guides Personal Computer (PC)

PHONAK

life is on

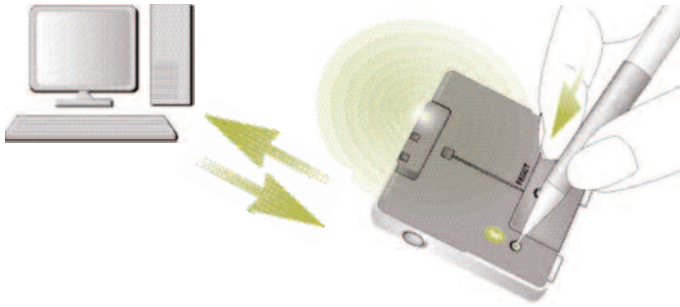

Pairing and connecting iCom with a Personal Computer (PC)

If you wish to use your iCom to listen to audio on your PC, you can do so if your PC supports Bluetooth.

It is necessary first to perform the Bluetooth pairing procedure between your iCom and PC. You should therefore refer to your PC's user guide or dealer support if you cannot complete the pairing procedure with the generic steps described here.

To pair your iCom and a Personal Computer (PC):

1. Charge and turn ON your iCom. Turn on your PC. Place your iCom near your PC.
2. On the PC, open the "Control Panel" and select the Bluetooth device wizard. This will assist you in setting up a Bluetooth connection between your iCom and PC.
3. Ensure the Bluetooth connectivity is turned ON: Either activate built-in Bluetooth functionality or connect the external Bluetooth adapter on your PC.
4. Initiate the pairing process on your iCom by pressing the Bluetooth button for about 2 seconds until the Bluetooth indicator flashes blue. Your iCom will remain in the pairing mode for two minutes or until the pairing is completed.
5. On the PC, choose to find and add a specific Bluetooth device.
6. Of the devices found, choose iCom.
7. When the PC prompts for a passcode or PIN, enter "0000" (four zeros).
8. Allow devices to complete the establishment of a trusted link.
9. If prompted on the PC to choose Bluetooth services, you may have several options available. Select the "Stereo Headset" functionality to stream audio in stereo. After choosing which services you wish to enable, you can finish the connection and exit the setup on the PC. Confirmation of successful connection is indicated on iCom by the Bluetooth indicator slowly flashing blue.