



Pairing and connecting iCom with your Samsung mobile phone

**1. Charge and turn on both mobile phone and iCom.
Place them next to each other.**

2. Prepare your mobile phone for pairing

Find the connectivity settings of your phone.
With most Samsung models one of the following navigation options applies.

Menu ▷ Settings ▷ Connectivity tab ▷ Bluetooth ▷ My devices

or

Menu ▷ Applications ▷ Bluetooth ▷ My devices

3. Prepare your iCom for pairing

Initiate the pairing process on your iCom by pressing the Bluetooth pairing button for about 2 seconds until the Bluetooth indicator flashes blue. Your iCom will remain in pairing mode for two minutes or until the pairing is completed.

4. Pair iCom and mobile phone

On your mobile phone, still in the Bluetooth menu, select

Search new device

(If you have not activated the Bluetooth functionality, you may be asked if you wish to turn on Bluetooth. Doing so will start the search process for Bluetooth devices.)

Your mobile phone should present a list of Bluetooth devices discovered.

Select "iCom" from this list.

Enter passcode "0000" (four zeros).

iCom will be added to the Paired devices list.

5. Connect iCom with mobile phone

Typically, iCom will be automatically connected after pairing. To connect manually, in the My devices menu, select iCom and select

Options ▷ Connect

On some phones you can configure to "Authorize device". Choosing so allows iCom to connect automatically without confirmation. We recommend that you enable this feature if available.

If you have difficulties navigating to the Bluetooth settings, please refer to the mobile phone's user manual or ask the mobile phone dealer.