



## Pairing and connecting iCom with your Sony Ericsson mobile phone

**1. Charge and turn on both mobile phone and iCom.  
Place them next to each other.**

### 2. Prepare your mobile phone for pairing

Find the connectivity settings of your phone.  
With most Sony Ericsson Models one of the following navigation options applies.

Menu ▷ Settings ▷ Connectivity tab ▷ Bluetooth ▷ My devices

or

Menu ▷ Connectivity ▷ Bluetooth ▷ My devices

or

(Tools or Main Menu) ▷ Control panel ▷ Connections ▷  
Bluetooth Devices tab

### 3. Prepare your iCom for pairing

Initiate the pairing process on your iCom by pressing the Bluetooth pairing button for about 2 seconds until the Bluetooth indicator flashes blue. Your iCom will remain in pairing mode for two minutes or until the pairing is completed.

### 4. Pair iCom and mobile phone

On your mobile phone, still in the Bluetooth menu, select **New device** or **Add**

(If you have not activated the Bluetooth functionality, you may be asked if you wish to turn on Bluetooth. Doing so will start the search process for Bluetooth devices.)

Your mobile phone should present a list of Bluetooth devices discovered.

Select **"iCom"** from this list.

Enter passcode **"0000"** (four zeros).

iCom will be added to the My devices list.

### 5. Connect iCom with mobile phone

Still on your mobile phone, in the My devices menu, select iCom and select **Connect**.

On some phones you can allow automatic connection. Choosing so allows iCom to connect automatically without confirmation. We recommend that you enable this feature if available.

If you have difficulties navigating to the Bluetooth settings, please refer to the mobile phone's user manual or ask the mobile phone dealer.