

# Audéo

## Product information



Phonak introduces a new product category with Audéo – Personal Communication Assistant. Discreet and stylish, it is designed for potential hearing instrument users with mild to moderate hearing losses who reject current products because of negative perceptions that hearing instruments are an obvious sign of old age and infirmity. Audéo employs industry

leading features, including CrystalSound for optimum speech clarity and digital SurroundZoom for improved understanding in noisy settings. Audéo is available in three performance/price levels and, with a range of domes and tip options, accommodates up to a moderately severe hearing loss.

### Audéo IX

- CrystalSound
- AutoPilot 4 program automatic
- Up to 5 manual programs
- Bionic Perception Processing strategy (BPP)
- 20 channel system
- Advanced digital Feedback Phase Inverter with NoWhistle Technology
- SoundRelax
- Adaptive digital SurroundZoom
- EchoBlock
- Real Ear Sound
- Microphone Protector
- Wind Noise Suppressor
- High Resolution Noise Canceler
- DataLogging<sup>plus</sup>
- Remote control optional
- User Preference Tuning
- MyLink FM compatible
- 312 battery
- Self Learning

### Audéo V

- TriPilot 3 program automatic
- Up to 5 manual programs
- dWDRC signal processing strategy
- 16 channel system
- Digital Feedback Phase Inverter
- Adaptive digital AudioZoom
- Microphone Protector
- Fine Scale Noise Canceler
- DataLogging<sup>plus</sup>
- Remote control optional
- User Preference Tuning
- MyLink FM compatible
- 312 battery

### Audéo III

- SoundManager with automatic feature activation
- No manual programs
- dWDRC signal processing strategy
- 6 channel system
- Feedback Phase Inverter
- AudioZoom
- Microphone Protector
- Digital Noise Canceler
- DataLogging
- 312 battery

#### Fitting software


- iPFG version 2.1 or higher, NOAH compatible

#### Hardware

- PC (IBM compatible) with NOAHlink and/or HI-PRO interface

## Programming

### Connecting the programming cable to an Audéo

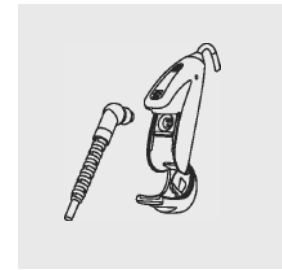




1. Open the battery compartment.




2. Plug the programming plug into the socket.  
Use the programming cable CS-44 A (Phonak version) only.

## Key features

The complete range of data sheets is available on [www.phonak.com](http://www.phonak.com)

		Audéo IX	Audéo V	Audéo III
				
<b>Performance profile</b>				
Maximum Power Output (dB SPL)	2 cc coupler	109	109	109
	Ear Simulator	121	121	121
Maximum gain (dB)	2 cc coupler	44	44	44
	Ear Simulator	56	56	56
Frequency range - Ear Simulator (Hz)		<100 Hz - 8300 Hz	<100 Hz - 8300 Hz	<100 Hz - 8300 Hz
Battery size		312	312	312
Working current (mA)		1.0 mA	1.0 mA	1.0 mA
CrystalSound		✓		
digital SourroundZoom (dSZ)		✓		
digital AudioZoom (dAZ)			✓	
AudioZoom (AZ)				✓

## Fitting ranges

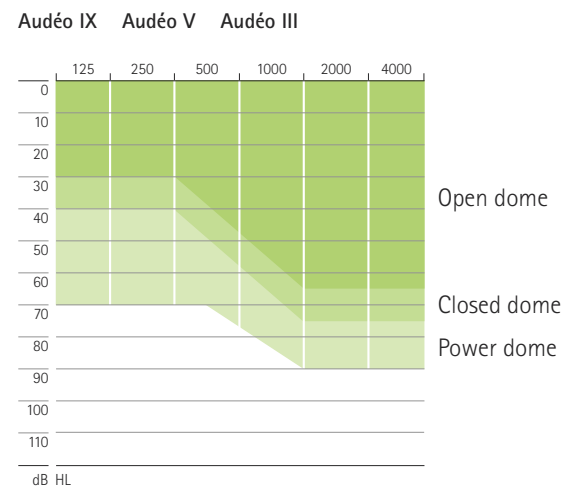





Mild to moderately severe or steeply sloping hearing loss.

## Technical description

- 1 Miniature dual microphone with protector
- 2 Battery compartment with ON/OFF switch
- 3 Serial number (inside battery compartment)
- 4 xReceiver Unit
- 5 Dome












## Options

	Audéo IX	Audéo V	Audéo III
xReceiver Standard Units (4 different lengths for left and right: xS 0 - xS 3)	S	S	S
 Open dome (3 different sizes: 5/7/10, two lengths)	0	0	0
 Closed dome (two lengths)	0	0	0
 Power dome (3 different sizes: 9/10/11, two lengths)	0	0	0
 SlimTip	0	0	0
Remote control compatibility	S	S	-
T-Coil	S	S	-
<b>Choice of 15 colors:</b> Back in Black (84), Classic Caddy (92), Lunar Eclipse (85), Snowblade (93), Precious Metal (86), Royal Velvet (79), Creme Brulee (89), Antique Mahogany (87), Raku Glaze (91), Fiery Temper (90), Flower Power (88), Solar Flare (82), Pure Passion (83), Pinot Noir (80), Green with Envy (81)	0	0	0

**Note:**  
Some options are country specific.  
Please refer to your local order form.

S = Standard 0 = Optional

## Accessories

	Audéo IX	Audéo V	Audéo III
<b>Remote controls</b>			
 WatchPilot2	✓	✓	-
 SoundPilot2	✓	✓	-
 KeyPilot2	✓	✓	-
<b>Multi-frequency receivers</b>			
 MyLink	✓	✓	-
<b>Multi-frequency transmitters</b>			
 inspiro	✓	✓	-
 SmartLink SX	✓	✓	-
 ZoomLink	✓	✓	-
 EasyLink	✓	✓	-
 Campus SX	✓	✓	-