



Instruments  
R Nios micro V  
L Nios micro V

Serial No.

## FM System Information for Teachers

### What is an FM system?

One of my biggest challenges is hearing when it is noisy, or when there is a large distance between myself and the person speaking. Even with my hearing instruments, noise, distance, a room's acoustics or lack of visual contact will make it more difficult for me to hear. FM systems overcome such difficult listening environments as the FM transmitter picks up speech signals at the source (for example your voice) and transmits them, clearly and without distortion, directly to my hearing instruments.

### The benefits of using FM technology

In the noisy and lively environment of school, FM helps me to learn and actively participate in class by making speech clearer. It "reduces" the distance between you and I in the classroom, and maintains speech intelligibility helping me to learn easily and efficiently. As listening will become so much easier, I will be able to focus for longer, making me less tired at the end of the day. Please use FM regularly as this is the biggest thing that you can do to help me learn in the classroom. I will hear you so much better which will make both our lives easier!

### Some tips when using FM

- Please use FM as much as possible so that I can hear your voice clearly at all times
- Please listen to my FM system every day before we use it to ensure it is working correctly
- If you don't want me to hear everything when you leave the classroom (e.g. when visiting the bathroom!), please remember to mute the microphone of the transmitter for as long as necessary

For more detailed information about FM, please visit the Phonak FM Inservice site [www.eSchoolDesk.com](http://www.eSchoolDesk.com)

## Details of my FM System

### FM Receiver

ML12i

Serial No.: (Right)

Serial No.: (Left)

### FM Transmitter

inspiro

Serial No.: