
Serenity SP

Gebrauchsanweisung
User Guide
Mode d'emploi


1

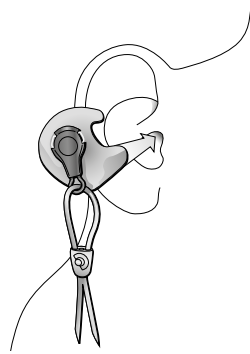
2



3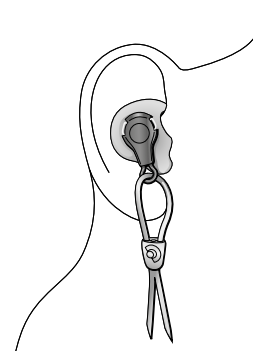

4

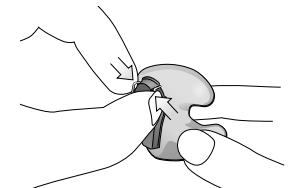5

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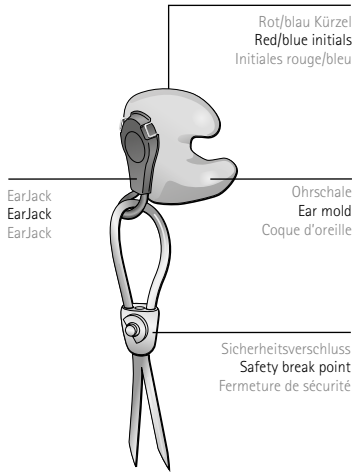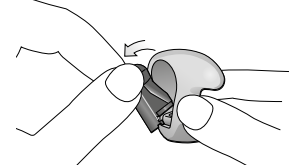Serenity SP 110WH, 105OR, 100YE
Professional hearing protection
Manufactured by Phonak Communications AG
Herrenschwandweg 4, 3280 Murten, Switzerland.

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Product Overview

Serenity SP hearing protection is custom adapted to the user's ear, is ultra-lightweight and is equipped with an acoustic filter, which attenuates surrounding noise.

The Serenity SP ear mold is a hollow shell manufactured from clinical nylon. It offers an unsurpassed level of comfort and is long-lasting, due to its outstanding resistance to chemical, mechanical and thermal effects.

An earJack™ adapter, which holds the acoustic attenuation filter, is clicked into the Serenity SP ear mold. This design guarantees a modularity unique to the Serenity product range; allowing

you to upgrade ear molds with communication headsets and/or dynamic protection filters.

Important User Information

- Put on, adjust, clean and maintain the hearing protection device in accordance with these instructions; otherwise, you may adversely impact the noise protection provided.
- The ear molds are reusable. They should be inspected regularly for serviceability.
- The material and design of the ear molds is long-lasting. Nevertheless, the shape of your ear canal may change over time (e.g., if your body weight changes significantly). This could adversely affect the sealing of the ear molds

and thereby the provided attenuation.

- For full protection, you must wear the hearing protection during all noise exposure.
- Certain chemical substances may be harmful to this product. Contact the manufacturer for more information.
- Hearing protection devices can block out warning sounds, alarms, and other important signals. Always pay extra attention to your surroundings when wearing hearing protection.
- Keep the clean ear molds in the pouch provided before and after use, as well as for transport. Store in a dry place at temperatures not below -20°C or above $+60^{\circ}\text{C}$.

Warning: Your protection should not be used where there is a risk that the connecting cords could be caught up during use.

- **Never use the device without having the earJack™ adapters correctly inserted into the custom-made ear molds. Verify that they are mounted according to these instructions before use.**

Instructions for Use

Putting the system on

- The ear mold marked red is for your right ear, the ear mold marked blue is for your left ear.
- Insert the ear mold into your outer ear with

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the opening pointing into your ear canal. **1**

- Twist the ear mold slightly to insert it fully into your ear. **2** Ensure that the device holds firmly in your outer ear and forms a good seal. **3**

Removing the system

- Hold the ear mold by the earJack and withdraw it from your ear carefully, using a slow forward movement to avoid a rapid pressure drop.

Maintaining the system

- Dismount the earJack™ adapter from the ear mold by pressing the snap lock springs together **4** and withdrawing the adapter, starting at the top. **5**

- Clean and disinfect the ear molds with soap and water. Never use water or other liquid to clean the earJack™. Let ear molds dry before use.
- Remount the earJack™ adapters into the ear molds by first hooking the lower hinge of the earJack™ adapter into the mold, **6** then pushing the upper part firmly into the mold until it clicks. **7** By correctly mounting the adapter, you will ensure a tight seal and proper operation of the hearing protection device.

Technical Data

Attenuation values for Serenity SP hearing protection are shown in the tables above.

Key

H | High-frequency attenuation value
M | Medium-frequency attenuation value
L | Low-frequency attenuation value
SNR | Single Number Rating attenuation value
APV | Assumed Protection Value

Conformity

Phonak Serenity SP hearing protection complies with Regulation (EU) 2016/425 for personal protective equipment and has been tested and approved according to EN 352- 2:2002. EC type examination certificate was issued by notified body BSI Group The Netherlands B.V. (2797), Say Building, John M. Keynesplein 9, 1066 EP, Amsterdam, Netherlands.

This hearing protection system also protects you against very short 'impulse' noises, such as gun shots, blasts and crashes. Nevertheless the degree of protection achieved depends on the energy of the noise in question and how often you are exposed to this noise.

For more information on protection against impulse noise, please visit www.phonak-communications.com or contact us: **Phonak Communications AG**, Herrenchwandweg 4, CH-3280 Murten.

Warning

Please note that some parts of Serenity SP are very small. You should store them safely out of children's reach.

Warranty

According to «General conditions of sales and delivery», the warranty period is two years from the date of purchase. Any responsibility for incidental or consequential damages will be refused.

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