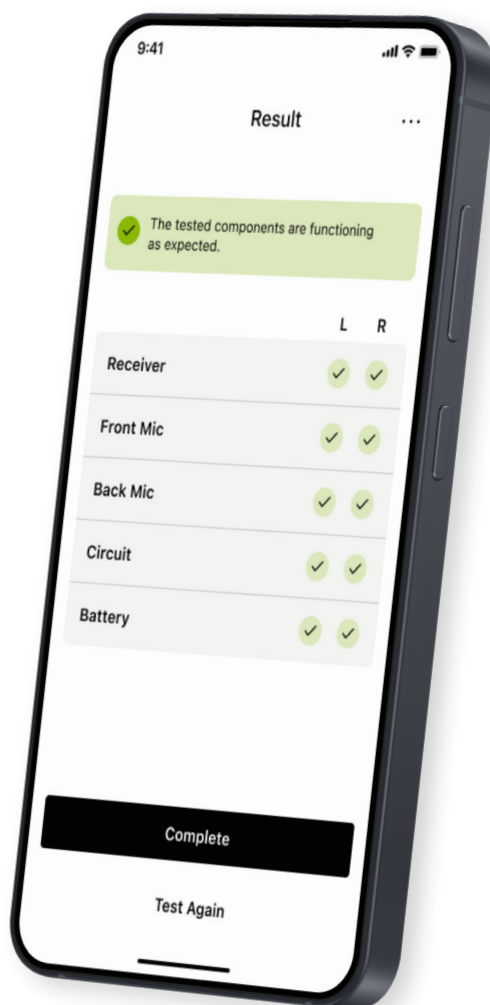
PhonakPro Check.

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← Privacy & Terms

Privacy Policy

Sonova processes your personal data for purposes such as making sure the PhonakPro Check app works correctly and securely. Your rights regarding the processing of your personal data include the right to access, rectify or request the deletion of your data, within the limits of the applicable law. Our Privacy Notice gives you more information about your personal data and how we use it.

[Read our Privacy Policy](#)

Terms & Conditions

Sonova offers PhonakPro Check in accordance with our Terms & Conditions.

[Read our Terms & Conditions](#)

I consent to the collection and processing of my personal data for the purpose of utilizing the PhonakPro Check software.

I understand that this app is intended to collect and process the personal information of my clients, and agree to collect their freely given and specific consent in line with the Client Privacy Notice, and applicable law.

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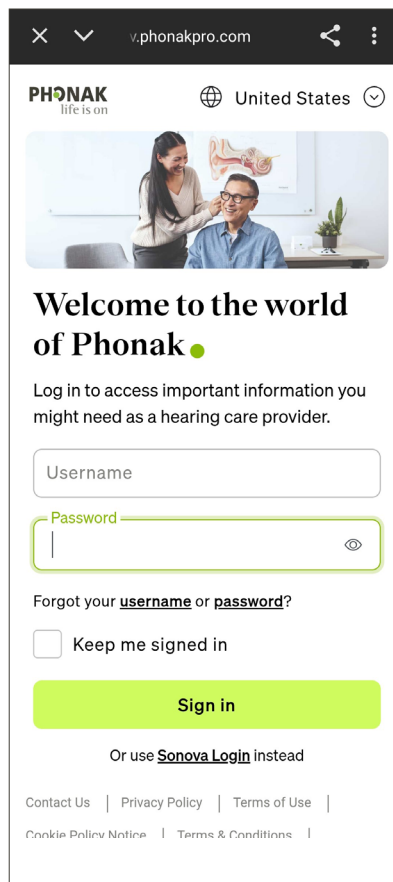
United States

Welcome to the world of Phonak

Log in to access important information you might need as a hearing care provider.

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	L	R
Front Mic	✗	✓
Back Mic	✓	✓
Receiver	✓	✓
Battery	✗	✗
Circuit	✓	✓

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SERIAL #	PRODUCT	REASON
2127N2TWW	Audeo L90-R	[Output resp Too weak, Rechargeable battery] Select...

PHONAK
life is on

Phonak Hearing Aid diagnostic report

	RIGHT	LEFT
Model	Audeo L90-RL	Audeo L90-RL
Serial number	2211H0C02	2211H0C03

	RIGHT	LEFT
Receiver	Pass	Pass
Front Microphone	Pass	Pass
Back Microphone	Pass	Pass
Battery	Battery Health within specifications	Battery Health within specifications

The tested components are functioning as expected.

Disclaimer: This report summarizes the results of a diagnostic assessment performed by PhonakPro Check on the connected hearing aids. It is intended for use by qualified hearing care professionals to support device evaluation and is not intended to provide information about the operational status of the device at the time of testing. The report does not constitute a medical diagnosis or assessment of the hearing aid wearer's hearing health. While the PhonakPro Check is designed to deliver accurate results, it may not detect all potential issues. Clinical judgment should be exercised, and further evaluation may be necessary.
© Sonova AG 2025

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Logo Apple App Store jest znakiem towarowym firmy Apple Inc.
Logo sklepu Google Play jest znakiem towarowym firmy Google LLC.
Sonova AG · Laubisrütistrasse 28