

Phonak


Reference Sheet

Using RogerDirect™ in a Phonak hearing aid

Important: It is recommended to keep a minimum of 3m/9.8ft between each Roger Installer during installation/uninstallation and to keep all other hearing aids near Roger Installer turned off. It is recommended to have Roger devices in the vicinity of Roger Installer turned off during an installation.

Installing Roger™ receiver into a Phonak hearing aid from Roger X

- Plug the Roger X into the Roger Installer (3).
- Turn on the Roger Installer (1) (the indicator light (2) will turn solid green).
- Indicator light (4) turns solid green (Roger X is ready for installation).
- Turn on the hearing aid and place it on the hearing aid logo.
- Wait for the indicator light (5) to blink green (installation is in progress) when indicator light turns solid green, installation is complete.
- Turn off the Roger Installer. Remove and store the Roger X. The hearing aid is ready for use with RogerDirect.

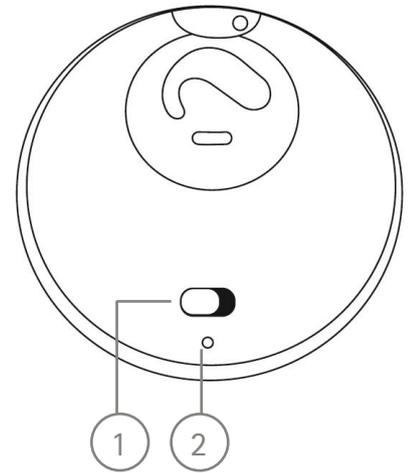

Uninstalling Roger receiver from a Phonak hearing aid to Roger X

- Plug the Roger X into the Roger Installer (3).
- Turn on the Roger Installer (1) (the indicator light (2) will turn solid green).
- Indicator light (4) remains inactive for several seconds (Roger X is ready to be uninstalled).
- Turn on the hearing aid and place it on the hearing aid logo.
- Wait for the indicator light (5) to turn solid green (confirms Roger is installed in this hearing aid).
- When uninstallation starts the indicator light (5) will blink green until the process is complete.
- Indicator light (4) turns solid green when uninstallation is complete (Roger is now installed into Roger X).
- Turn off the Roger Installer. Remove the Roger X. The hearing aid can no longer be used with Roger.

