


Phonak

Fast Fact

Love at first sound: The Marvel pre-calculation comes to non-wireless Virto models

What is it?

The Marvel pre-calculation is a special gain setting designed for first-time hearing aid users that slightly reduces the gain of the hearing aid above 3 kHz. This reduction in gain is designed to make adjusting to amplification easier for the client while providing exceptional sound quality from the first fit for "love at first sound." With many first-time hearing aid users desiring a custom hearing aid, the Marvel pre-calculation is especially valuable for those wearing Virto M-312 NW O, Virto M-10 NW O, and Virto M-Titanium.

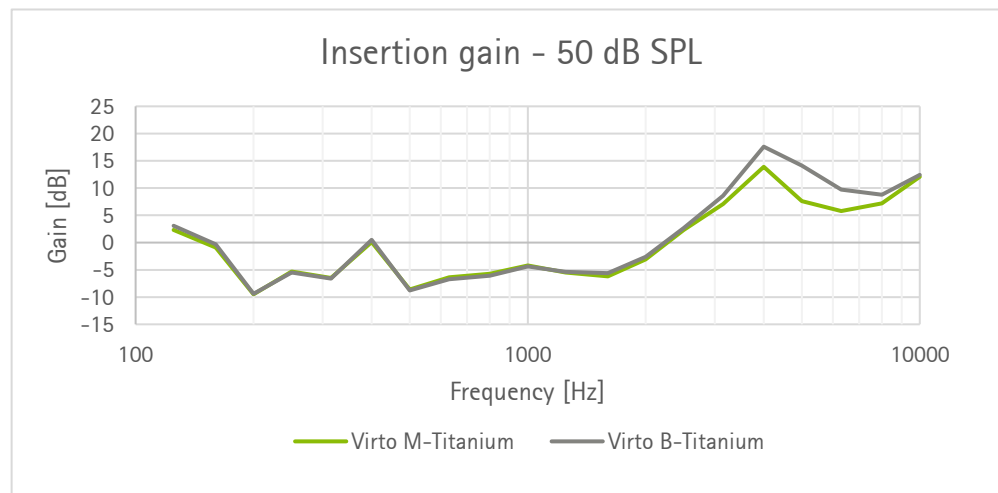

Why use it?

With the Marvel first-fit pre-calculation, spontaneous acceptance of the hearing aids is improved while maintaining audibility in complex listening environments.¹

Real-ear aided gain measures show the difference in output between Virto M-Titanium with the Marvel pre-calculation versus Virto B-Titanium, both set to 80% target gain.

How do I use it?

There are two ways to activate the first-fit pre-calculation in Virto Marvel:

- When connecting the hearing aids in Phonak Target for the first time, change the suggested client experience level under **[Optimized for]** to **[First-time user]**
- Under **[Global tuning]**, open the drop-down menu under **[Gain level]** and select **[80% target gain]**

For more information, go to www.phonakpro.com or contact your Phonak representative.

¹Rakita, L., Stewart, E., & Drexler, J. (2018). Best first fit experience from Phonak: Phonak Marvel. *Phonak Field Study News*. Retrieved from www.phonakpro/evidence. Accessed February 19, 2020.