

Lyric



SoundLync
Quick
Reference



ON 1 beep (remove)

SLEEP 2 beeps (remove)

OFF 2 beeps (hold)
4 beeps

PHONAK

Lyric



SoundLync
Quick
Reference

VOLUME

2 beeps (remove for 2 sec)

1 beep (hold)

Volume increase starts 1 step louder

Each step: louder beep

Maximum setting: double beep

Note: Lyric is muted during volume adjustment

0318-1220-0 2020.01.14-0.0/shw/Printed In XXXXX © Phonak AG/All rights reserved

PHONAK