

# Phonak Audéo™ P

## Product information



Phonak Audéo P is the comprehensive RIC portfolio for mild to profound hearing losses. Phonak Audéo P includes five designs, four performance levels and five external receivers, including the new ActiveVent™ receiver.

Powered by the innovative technology of the Phonak Paradise platform, Phonak Audéo P hearing aids are the solution for those clients that prefer flexible and discreet RIC housings.

For more information visit [www.phonakpro.com](http://www.phonakpro.com)

A Sonova brand

**PHONAK**  
life is on

# Product description



		Audéo P-312 Audéo P-312 Trial P90/P70/P50/P30	Audéo P-13T Audéo P-13T Trial P90/P70/P50/P30	Audéo P-R Audéo P-R Trial P90/P70/P50/P30	Audéo P-RT Audéo P-RT Trial P90/P70/P50/P30	Audéo P-RL Audéo P-RL Trial P90/P70/P50/P30
Battery size		312	13			
Multi-function button		•	•	•	•	•
Telecoil			•		•	
Bluetooth®		•	•	•	•	•
Hydrophobic coating		•	•	•	•	•
IP rating		IP68 <sup>1</sup>	IP68 <sup>1</sup>	IP68 <sup>1</sup>	IP68 <sup>1</sup>	IP68 <sup>1</sup>
Dimensions	L x W x D	26.8 x 11.2 x 7.1 mm (1.05 x 0.44 x 0.28")	29.0 x 11.9 x 8.1 mm (1.13 x 0.47 x 0.32")	28.0 x 11.9 x 8.0 mm (1.11 x 0.47 x 0.32")	30.5 x 12.5 x 8.0 mm (1.20 x 0.49 x 0.32")	30.0 x 13.1 x 8.8 mm (1.18 x 0.52 x 0.35")
	Weight	1.1 g (0.04 oz)	1.4 g (0.05 oz)	2.1 g (0.07 oz)	2.4 g (0.08 oz)	2.75 g (0.97 oz)
Charging times <sup>2</sup>	0% to 100%			3 h	3 h	3 h
	0% to 80%			1.5 h	1.5 h	1.5 h
	30 min. fitting			10 min.	10 min.	10 min.
<b>S Receiver</b>						
Max. Power Output (dB SPL)	2 cc coupler	111	111	111	111	111
Max. gain (dB)	2 cc coupler	47	47	47	47	47
Frequency range (Hz)		<100 - >8000	<100 - >8000	<100 - >8000	<100 - >8000	<100 - >8000
Working current (mA)		1.4	1.4			
<b>M Receiver</b>						
Maximum Power Output (dB SPL)	2 cc coupler	114	114	114	114	114
Max. gain (dB)	2 cc coupler	51	51	51	51	51
Frequency range (Hz)		<100 - >8000	<100 - >8000	<100 - >8000	<100 - >8000	<100 - >8000
Working current (mA)		1.4	1.4			
<b>MAV Receiver</b>						
Maximum Power Output (dB SPL)	2 cc coupler			119	119	119
Max. gain (dB)	2 cc coupler			50	50	50
Frequency range (Hz)				<100 - >8000	<100 - >8000	<100 - >8000
Working current (mA)						
<b>P Receiver</b>						
Maximum Power Output (dB SPL)	2 cc coupler	122	122	122	122	122
Max. gain (dB)	2 cc coupler	59	59	59	59	59
Frequency range (Hz)		<100 - 6300	<100 - 6300	<100 - 6300	<100 - 6300	<100 - 6300
Working current (mA)		1.5	1.5			
<b>UP Receiver</b>						
Max. Power Output (dB SPL)	2 cc coupler	132	132	132	132	132
Max. gain (dB)	2 cc coupler	71	71	71	71	71
Frequency range (Hz)		<100 - 6100	<100 - 6100	<100 - 6100	<100 - 6100	<100 - 6100
Working current (mA)		1.4	1.4			

# Housing colors



**P1**  
Sand Beige



**HO**  
Beige



**P3**  
Sandalwood



**P4**  
Chestnut



**P5**  
Champagne



**P6**  
Silver Gray



**P7**  
Graphite Gray



**P8**  
Velvet Black

<sup>1</sup> IP68 indicates that the hearing aid is water and dust resistant. It survived continuous immersion in 1 meter of fresh water for 60 minutes and 8 hours in a dust chamber as per the IEC60529 standard.  
<sup>2</sup> Actual charging time depends on the remaining battery life, but it will not exceed 3 hours.

## Performance levels

	Premium	Advanced	Standard	Essential	Features	Premium	Advanced	Standard	Essential
<b>AutoSense OS™ 4.0</b>	Premium	Advanced	Standard	Essential	UltraZoom	•	•	•	•
Comfort in Echo	•				Dynamic Noise Cancellation	•			
Speech in Car	•				Speech Enhancer	•			
Speech in Loud Noise	•				EchoBlock	•			
Music	•	•			Tap control <sup>a)</sup>				
Comfort in Noise	•	•	•		- Siri/Google Assistant	•	•		
Speech in Noise	•	•	•	•	- Pause/resume streaming	•	•		
Calm Situation	•	•	•	•	- Accept/end phone call	•	•		
Media speech + mic	•	•	•	•	StereoZoom	•	•		
Media music + mic	•	•	•	•	WindBlock	•	•		
<b>Additional programs</b>					SoundRelax	•	•		
Max. additional programs	4	4	4	4	Motion Sensor Hearing <sup>a)</sup>	•	•	•	
Comfort in Echo	•				DuoPhone	•	•	•	
Speech in Loud Noise	•	•			Real Ear Sound	•	•	•	
Speech in 360°	•	•			Roger and Directional setting	•	•	•	
Speech in Noise	•	•	•	•	SoundRecover2	•	•	•	•
Calm Situation	•	•	•	•	WhistleBlock	•	•	•	•
Comfort in Noise	•	•	•	•	NoiseBlock	•	•	•	•
Music	•	•	•	•	RogerDirect™	•	•	•	•
Acoustic phone	•	•	•	•	User Preference Tuning	•	•	•	•
Custom program	•	•	•	•	QuickSync	•	•	•	•
<b>Streaming programs</b>					AOV	•	•	•	•
RogerDirect + mic	•	•	•	•	Tinnitus Balance	•	•	•	•
PartnerMic + mic	•	•	•	•	auto Acclimatization	•	•	•	•
Phone call + mic	•	•	•	•	Environmental balance	•	•	•	•
					Fine tuning channels	20	20	16	12


<sup>a)</sup> Only available for Phonak Audéo P-R, RT and RL

## Fitting

	Audéo P-312 Audéo P-312 Trial	Audéo P-13T Audéo P-13T Trial	Audéo P-R Audéo P-R Trial	Audéo P-RT Audéo P-RT Trial	Audéo P-RL Audéo P-RL Trial
Phonak Target	7.0 or higher	7.0 or higher	7.2 or higher	7.2 or higher	7.2 or higher
Noahlink Wireless	•	•	•	•	•
NOAHLink		•			
HI-PRO, HI-PRO2		•			

# Acoustic coupling

Length	S Receiver	M Receiver	MAV Receiver	P Receiver	UP Receiver
Available in four different lengths (0, 1, 2, 3), left and right	•	•	•	•	•
<b>Dome</b>					
Cap dome	•	•		•	
Open dome Available in three sizes (S, M, L)	•	•		•	
Vented dome Available in three sizes (S, M, L)	•	•		•	
Power dome Available in three sizes (S, M, L)	•	•		•	
<b>SlimTip*</b>					
Acryl	•	•		•	
Silicone	•	•		•	
Titanium AV			•		
Universal AV Available in four shapes (A, B, C, D)			•		
<b>cShell</b>					
Acryl	•	•		•	•
Titanium	•	•		•	•
<b>Fitting ranges</b>					

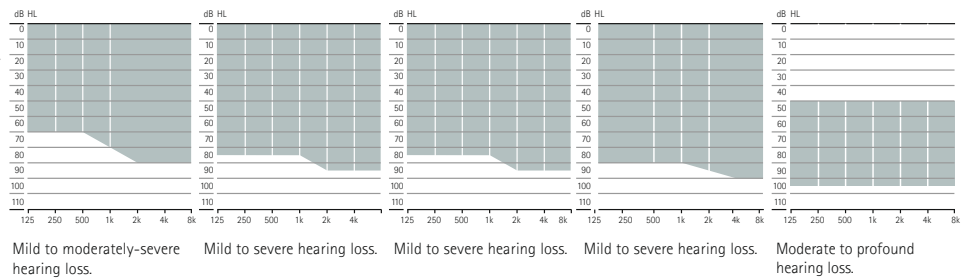

## ActiveVent™ Receiver

The brand new ActiveVent Receiver (MAV) offers the benefit of both open and closed fit in one receiver. It is compatible with all rechargeable Audéo P devices. ActiveVent receivers can be instantly fit for in-clinic demonstration or short term home trial, with a Universal SlimTip AV that is available in 4 generic shapes. A SlimTip Titanium AV that is custom made for the clients ear is recommended for long term wear and to achieve optimal acoustic performance.

## Phonak Audéo P Trial™

The Phonak Audéo P Trial hearing aids can be quickly set to the required performance level in order to suit your recommendation. All levels of technology in one Phonak Audéo P Trial hearing aid provides you with many options for providing the hearing solution at the first appointment or bridging the gap during repairs.

\*Available with extra canal length option

## Phonak Audéo P charging options



### Phonak Charger Case Combi

The Phonak Charger Case Combi is a charger, drying kit and protective hard case all in one, offering a robust and splash-proof housing.



### Phonak Power Pack

For charging rechargeable hearing aids on the go, the Phonak Power Pack can be easily attached to the bottom of the Phonak Charger Case Combi.



### Phonak Charger Case Go and Phonak Life Charger

There are two options available for Audéo P-RL. Phonak Charger Case Go features a built-in battery that can charge two hearing aids up to three times.

## Phonak Audéo P Bluetooth® versions

Direct calls/audio streaming

Bluetooth® 4.2 wireless technology and most older Bluetooth phones

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Sonova AG is under license.

## Wireless communication portfolio

	Audéo P-312 Audéo P-312 Trial P90/P70/P50/P30	Audéo P-13T Audéo P-13T Trial P90/P70/P50/P30	Audéo P-R Audéo P-R Trial P90/P70/P50/P30	Audéo P-RT Audéo P-RT Trial P90/P70/P50/P30	Audéo P-RL Audéo P-RL Trial P90/P70/P50/P30
<b>Wireless accessory</b>					
TV Connector	•	•	•	•	•
Phonak PartnerMic	•	•	•	•	•
Phonak RemoteControl	•	•	•	•	•
<b>Roger</b>					
RogerDirect™	•	•	•	•	•
<b>Apps</b>					
myPhonak	•	•	•	•	•

For more information please visit [www.phonakpro.com/audeoparadise](http://www.phonakpro.com/audeoparadise)

For information on Roger microphones please visit [www.phonakpro.com/roger](http://www.phonakpro.com/roger)

For more information on eSolutions please visit [www.phonakpro.com/smartapps](http://www.phonakpro.com/smartapps)

# life is on


At Phonak, we believe that well-hearing equates to well-being and is essential to living life to the fullest. For more than 70 years, we have remained passionate about creating a world where 'life is on' for everyone. Our innovative hearing solutions are designed for people of all ages and all degrees of hearing loss, to connect socially, thrive mentally and emotionally.

[www.phonak.com](http://www.phonak.com)



027-0602-02/15.00/2022-01/na © 2022 Sonova AG All rights reserved



 Sonova AG · Laubisrütistrasse 28  
CH-8712 Stäfa · Switzerland

