

Phonak CROS P

Product information



Phonak CROS P is a solution for unilateral hearing loss designed to help the wearer follow* conversations wherever they come from**¹. CROS P is worn on the unaidable ear to pick up sounds and wirelessly transmits them to a Phonak Audéo™ P hearing aid worn on the ear with better hearing. CROS P has no acoustical output.

CROS P includes two models and is compatible with Audéo P-13T, Audéo P-R and Audéo P-RT hearing aids in all performance levels. For more information visit www.phonakpro.com

* Follow - means awareness of conversation for the wearer to attend to.



** when using a CROS system compared to unaided.

¹ Stewart, E. & Woodward, J. (2021). Out of the (Head) Shadow: A Systematic Review of CROS/BiCROS Literature. The Hearing Review, 28(8), 22-25. Retrieved from <https://www.hearingreview.com/inside-hearing/research/head>, accessed August 9, 2021.

A Sonova brand

PHONAK
life is on

Product description

			
		CROS P-R	CROS P-13
Battery		rechargeable	size 13
Multi-function button		•	•
CROS balance		•	•
IP rating		IP68 ¹	IP68 ¹
Dimensions	L x W x D	28.0 x 11.9 x 8.0 mm (1.11 x 0.47 x 0.32")	29.0 x 11.9 x 8.1 mm (1.13 x 0.47 x 0.32")
	Weight	2.1 g (0.07 oz)	1.4 g (0.05 oz)
Charging times ²	0% to 100%	3 h	
	0% to 80%	1.5 h	
	30 min. fitting	10 min.	
Compatibility with hearing aid		Audéo P-R, Audéo P-RT	Audéo P-13T

Housing colors



Earpiece options

Standard dome (CROS Tube)	<ul style="list-style-type: none"> Standard dome available in three sizes (S, M, L) CROS Tube is available in left/right and four different lengths (0, 1, 2, 3)
SlimTip (CROS Tube)	<ul style="list-style-type: none"> Custom made earpiece CROS Tube is available in left/right and four different lengths (0, 1, 2, 3)
CROS Tip	<ul style="list-style-type: none"> Fully custom made earpiece for CROS, available in four different cable lengths (0, 1, 2, 3)

Phonak CROS P-R charger options

CROS P-R can be charged together with the paired Audéo P-R or Audéo P-RT using either a Phonak Charger Case Combi or a Phonak Mini Charger Case.

For more information please refer to the product information of Phonak Audéo P.

¹ IP68 indicates that the hearing aid is water and dust resistant. It survived continuous immersion in 1 meter of fresh water for 60 minutes and 8 hours in a dust chamber as per the IEC60529 standard.

² Actual charging time depends on the remaining battery life, but it will not exceed 3 hours.

Fitting

	CROS P-R	CROS P-13
Phonak Target	7.1.9 or higher	7.1.9 or higher
Noahlink Wireless	•	•
NOAHLink		•
HI-PRO, HI-PRO2		•

Wireless Communication Portfolio

When using wireless accessories, they will connect to the hearing aid on the better ear. When the signal from an accessory is present, the CROS signal will pause, and will resume as soon as the signal from the accessory has stopped. Please consult the product information of Phonak Audéo P for a complete list of compatible accessories.

Key features

CROS P can be fit with an Audéo P of any performance level. Therefore the set of features that a client wearing a CROS P system will experience is mainly dependent on the performance level of the Audéo P hearing aid. Please refer to the product information of Phonak Audéo P or additional information.

StereoZoom

StereoZoom can help improve speech intelligibility in challenging noisy environments¹. StereoZoom is available for hearing aids of Premium and Advanced performance levels.

CROS balance

CROS balance adjusts the loudness ratio between the CROS device and the hearing aid.

AutoSense OS™ 4.0

AutoSense OS 4.0 automatically senses your client's listening environment in real-time. It draws from multiple features, while accurately blending them to provide a seamless listening experience.

¹ Stewart, E., Rakita, L. & Drexler, J. (2019). StereoZoom Part 1: The benefit of wirelessly connected narrow directionality in Phonak hearing aids for speech intelligibility. Phonak Compendium, retrieved from www.phonakpro.com/evidence, accessed August 2021

CROS P performance

CROS P offers the Speech in Loud Noise program with StereoZoom. It is available in combination with hearing aids of Premium and Advanced performance levels. For the Standard and Essential performance levels, CROS P provides Real Ear Sound and omnidirectional microphone modes. On the better ear, CROS P clients fully benefit from all Paradise features.

Hearing aid performance level	Available microphone modes for CROS P		
	StereoZoom	Real Ear Sound	Omnidirectional
Premium	•	•	•
Advanced	•	•	•
Standard		•	•
Essential			•

For more information please visit www.phonakpro.com/cros-p

For more information on Audéo P hearing aids please visit www.phonakpro.com/audeoparadise

life is on

At Phonak, we believe that well-hearing equates to well-being and is essential to living life to the fullest. For more than 70 years, we have remained passionate about creating a world where 'life is on' for everyone. Our innovative hearing solutions are designed for people of all ages and all degrees of hearing loss, to connect socially, thrive mentally and emotionally.

www.phonakpro.com



027-0622-02\2.00\2021-09\ja © 2021 Sonova AG All rights reserved