

Phonak Drying kit.

The drying kit is packaged in a case containing a cup and a desiccant capsule.

The drying capsule is indicated for dehumidifying hearing aids in order to avoid breakdown due to moisture. It's a translucent plastic capsule of 30 mm in diameter and 12mm in height, which contains crystals of orange silica gel, and closed by a cardboard seal. It's packaged in an individual sealed bag.

The cup ensures a hermetic environment in order to guarantee efficient dehumidification of hearing aids thanks to the action of capsules. It has a foam disc to prevent damage to the hearing instruments and to keep them stable within the cup during the operation.

Properties

The drying kit is a maintenance product specially developed to effectively absorb moisture from hearing aids. It meets the requirements of Class I Medical Devices, according to the Regulation 2017/745/EU related to Medical Devices, while respecting environmental and safety requirements for health.

Using advice

- Place the capsule, face upwards, under the foam. Put the hearing aids in the foam with the battery compartment open.
- Close the cup and let the capsule act all the night. Remember to close the cup after each use.
- Change the capsule when it becomes almost white. The duration of use is about 4 months.
- Daily use is recommended, preferably at night. Regular dehumidification increases the life of the hearing aids and preserves their effectiveness.



Precautionary statements

- External use
- Do not pierce the capsule
- Do not swallow
- Keep out of reach children

Product composition

- Desiccant capsule: Silica gel
- Drying cup: Polypropylene