Care Guide.

Behind-The-Ear (BTE) Hearing Aid

EN | Click for your language

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ES | Haga clic para su idioma

PT | Clique para o seu idioma

Care Guide.

Behind-The-Ear (BTE) Hearing Aid

Content in kit

52 Cleaning wipes

1 Earmold blower

1 Cleaning spray

1 Microfiber cloth

3 Boxes of CeruStop filters
(a total of 24 wax filters)

1 Brush


1 Vent brush


2 Green vent
cleaning wires

☐ Coupling: earmold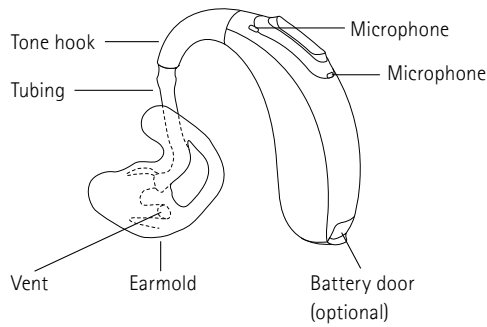

☐ Coupling: SlimTube with dome

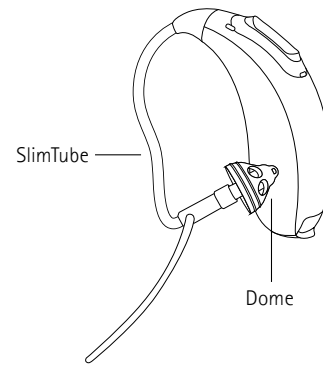Caring & cleaning

Congratulations on prioritizing your hearing health with your new hearing aids! While these devices are designed for long-lasting reliability and durability, it's essential to protect them through regular maintenance. Taking just a few minutes each day to care for your hearing aids will help ensure they continue to perform at their best.

Earwax is a natural secretion that helps protect your ears. However, too much earwax can cause issues for your hearing aids, like blocking sound or making it unclear. Most hearing aids have filters to keep wax out. It's important to check and change these filters regularly to ensure your hearing aids work well.

Your Care Guide offers you helpful steps for the proper maintenance of your hearing aids. Additionally, consider utilizing the included monthly calendar to assist in keeping track of your hearing aid maintenance.

Daily

Inspect your hearing aid before use daily to ensure it is clean. If needed, spray a tissue with the cleaning spray to wipe the earmold and hearing aid body. Remove any earwax accumulated at the tip of the dome or earmold. After, wipe the hearing aid using a clean and absorbent cloth or the included microfiber cloth.

Important: Never spray your hearing aids directly with the cleaning spray.


If you notice any debris gently brush the surface of the hearing aid or earmold. If you have a dome avoid using a brush but rather wipe it off with a clean cloth.

Important: Be careful not to brush into the microphone openings to avoid damaging them.

Weekly

Use a cleaning wipe to remove any wax or debris from the hearing aids. After, wipe the hearing aids using a clean and absorbent cloth or the included microfiber cloth.

If you have an earmold use the green vent cleaning wire or vent brush to clear the vent of any wax or debris. Thread the vent cleaning wire or vent brush through the vent and wipe the tip of the tool with a cleaning wipe.

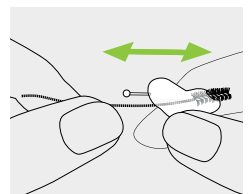Watch the instruction video on how to clean your hearing aid.

Monthly: SlimTube and dome maintenance

If you have a SlimTube with a dome, the SlimTube should be cleaned and the domes replaced every month to ensure optimal performance.

Important: Remove the dome gently, avoid pulling or applying excessive force to the SlimTube.

Replacement SlimTubes and Domes are not included in this kit. Please contact your hearing care professional if replacements are needed.


Watch the instruction video on how to clean SlimTubes and replace the dome.



1

Hold the speaker in one hand and the dome in the other.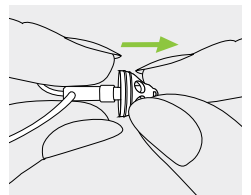



2

Gently pull off the dome to remove it.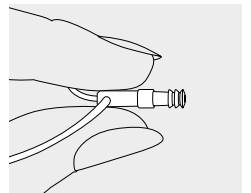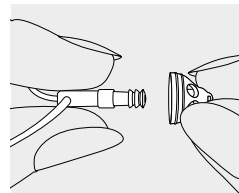



3

Unscrew the SlimTube from the hearing aid.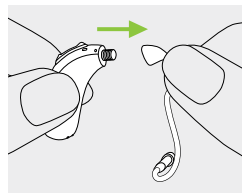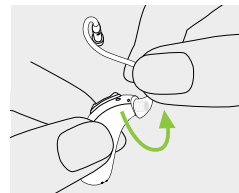


4

Thread a green vent cleaning wire through the SlimTube. You may have to temporarily straighten out the tubing to push it through. Wipe any ear wax off the end of the green wire with a cleaning wipe or tissue.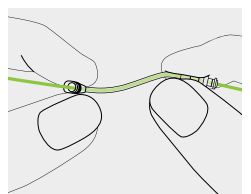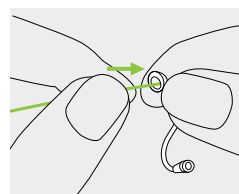



5

Pick up a new dome. Slide the dome over the SlimTube tip.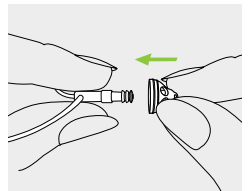

6

The speaker and the dome should fit perfectly together.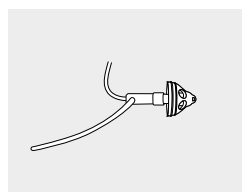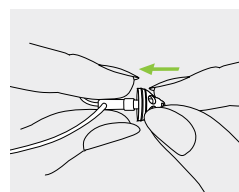

7

Screw the SlimTube back onto the hearing aid with a clockwise turn.

Monthly: Earmold tubing maintenance

If you have an earmold your tubing needs to be cleaned **monthly** or when:

- You observe that moisture is present inside the tubing.
- The hearing aid does not produce normal sound.

Tubing needs to be replaced regularly at least every 6 months. How often can depend on the individual. Check for:

- The tubing becoming less clear or yellow in color.
- The tubing becomes stiff and difficult to remove from the hearing aid.
- You notice increased feedback or whistling from your hearing aid.

See your hearing care professional for tubing replacements.


Watch the instruction video on how to inspect and clean your tubing.



1

First, gently remove the tubing from your hearing aid. Grip the tubing below the tone hook and pull it off.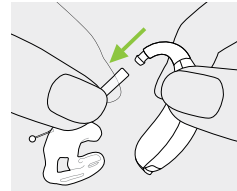

2

Insert the air blower into the end of the tubing that was connected to the hearing aid. Squeeze the blower to clear moisture and debris from the tubing. This can also be done for SlimTubes. Re-attach the tubing to the hearing aid when complete.

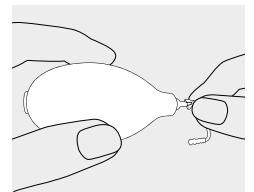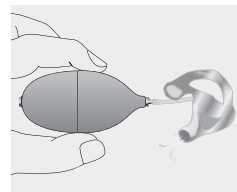Make sure you change your dome (if applicable) or clean your tubing at least monthly. Mark the calendar below when done.

JAN	FEB	MAR	APR	MAY	JUN
JUL	AUG	SEP	OCT	NOV	DEC

Looking for more support with your hearing aids?

Visit our support page at phonak.com/SupportHome

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