Care Guide.

In-The-Ear (ITE) Hearing Aid

EN | Click for your language

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ES | Haga clic para su idioma

PT | Clique para o seu idioma

Care Guide.

In-The-Ear (ITE) Hearing Aid

Content in kit

52 Cleaning wipes

1 Drying cup

6 Desiccant
(drying) capsules

6 Boxes of CeruStop filters
(a total of 48 wax filters)

1 Brush

1 Vent brush

2 Green vent
cleaning wires

1 Microfiber cloth


Caring & cleaning

Congratulations on prioritizing your hearing health with your new hearing aids! While these devices are designed for long-lasting reliability and durability, it's essential to protect them through regular maintenance. Taking just a few minutes each day to care for your hearing aids will help ensure they continue to perform at their best.


Earwax is a natural secretion that helps protect your ears. However, too much earwax can cause issues for your hearing aids, like blocking sound or making it unclear. Most hearing aids have filters to keep wax out. It's important to check and change these filters regularly to ensure your hearing aids work well.

Your Care Guide offers you helpful steps for the proper maintenance of your hearing aids. Additionally, consider utilizing the included monthly calendar to assist in keeping track of your hearing aid maintenance.


☐ ITE rechargeable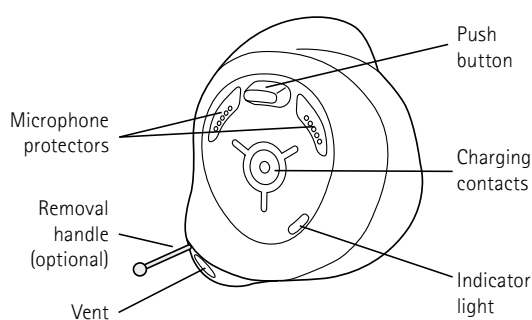

Side view

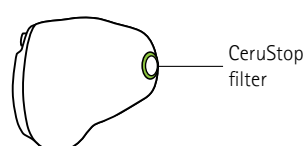CeruStop exchange tool


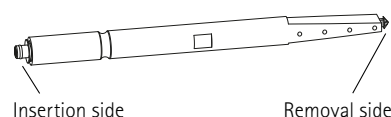
Daily – cleaning and drying (rechargeable)

Inspect your hearing aid before use daily to ensure it is clean. If needed, use the microfiber cloth to gently wipe the hearing aid.

Use the brush to gently brush away any visible debris from the microphone protectors and CeruStop filter.


Use the drying cup with a desiccant (drying) capsule to dry out the hearing aid at least once a week. Turn the hearing aid off before placing it in the drying cup by holding the push button until the light turns red. Leave the hearing aids in the drying cup for several hours (such as over night).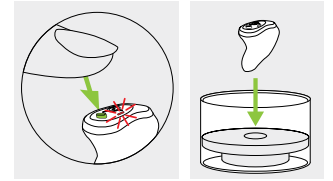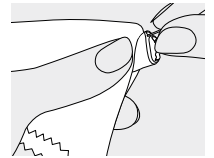

Weekly – vent cleaning (rechargeable)

Use a cleaning wipe to remove any wax or debris. After, wipe the hearing aids using a clean and absorbent cloth or the included microfiber cloth.

Use the vent cleaning wire or vent brush to clear the vent of any wax or debris by threading through the vent and wiping the wire or brush with a cleaning wipe.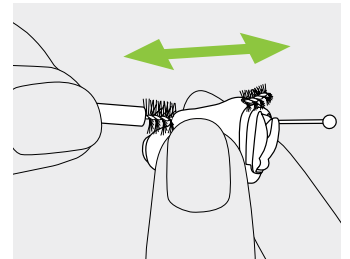

Watch the instruction video on how to clean your hearing aid.


Weekly – charger care (rechargeable)

Inspect the charger case and charger ports every other week for any build up of wax and debris. It is important to keep the charger case and the charger ports clean, to ensure a secure placement of the hearing aids and a reliable charge.


Use the microfiber cloth and brush provided to gently wipe and clean the charger case, especially the charger ports.

⚠ Always disconnect the charger from the power mains before cleaning.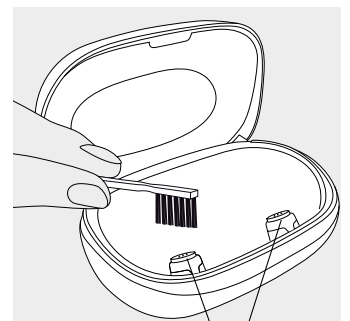


charger ports


□ ITE size 10 non-rechargeable battery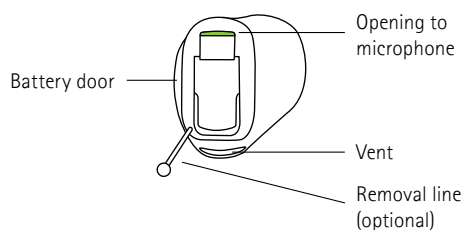


□ ITE size 312 non-rechargeable battery

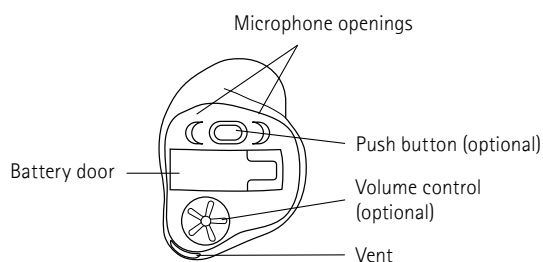Side view

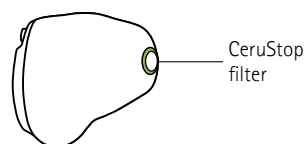CeruStop exchange tool


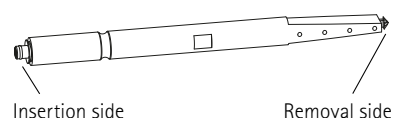
Daily – cleaning and drying (non-rechargeable)

Inspect your hearing aid before use daily to ensure it is clean. If needed, use the microfiber cloth to gently wipe the hearing aid.

Use the brush to gently brush away any visible debris from the microphones, CeruStop filter, and battery door.


Turn off the hearing aid at night by opening the battery door. Place the hearing aid in the drying cup with a desiccant (drying) capsule overnight.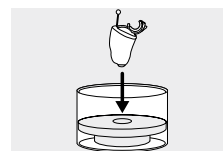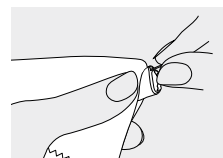

Weekly – vent cleaning (non-rechargeable)

Use a cleaning wipe to remove any wax or debris. After, wipe the hearing aids using a clean and absorbent cloth or the included microfiber cloth.

Use the vent cleaning wire or vent brush to clear the vent of any wax or debris by threading through the vent and wiping the wire or brush with a cleaning wipe.

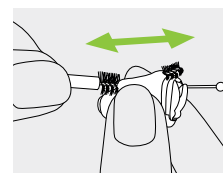Watch the instruction video on how to clean your hearing aid.

Replacing the CeruStop filter

Your CeruStop filter should be changed when:

- You observe that the CeruStop filter is obstructed.
- The hearing aid does not produce normal sound.
- At least every other week.


Watch the instruction video on how to change the wax filter.


1

Insert the removal side of the exchange tool into the used wax filter.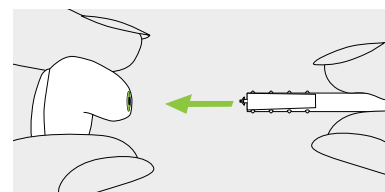


2

Carefully pull and remove the wax filter straight out of the hearing aid. Please do not twist the wax filter when removing.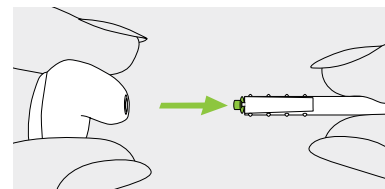


3

To insert the new wax filter, gently push the insertion side of the exchange tool straight into the hole.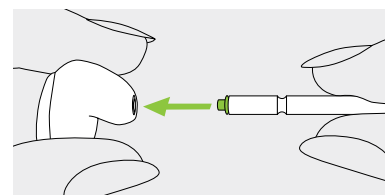

4


Pull the tool straight out. The new wax filter will remain in place.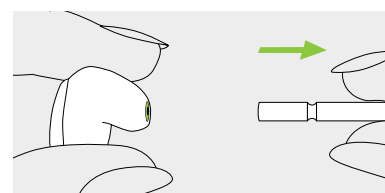


Replacing the non-rechargeable battery

Ensure you use the correct sized battery for your hearing aid. Your hearing aid may appear different than shown depending on configuration.

☐ ITE size 10 (yellow sticker)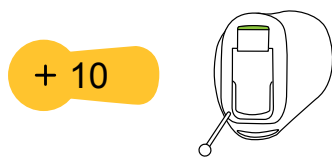

☐ ITE size 312 (brown sticker)

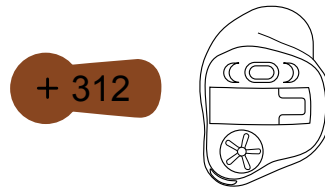
Watch the instruction video on how to replace your hearing aid battery.

1


Remove the sticker from the new battery. Wait 2 minutes to allow battery to activate.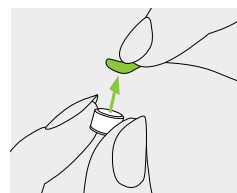

~ 2 minutes



2

Gently lift the battery door. It will swing open on a hinge. Remove the used battery.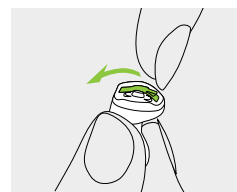

3

Place the new battery in the battery door. The positive (flat) side should be facing out.

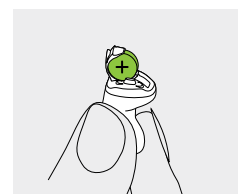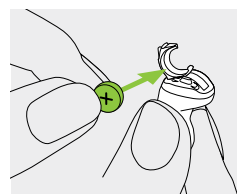4

Gently close the battery door. Your hearing aids will then power on.

The battery door should close easily. If excessive force is used the battery door may break. Check for any obstructions and that the battery is orientated correctly.

Mark the calendar below every other week to indicate when you changed your CeruStop filter.

JAN	FEB	MAR	APR	MAY	JUN
JUL	AUG	SEP	OCT	NOV	DEC

Looking for more support with your hearing aids?

Visit our support page at phonak.com/SupportHome

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