Care Guide.

Receiver-In-The-Canal (RIC) Hearing Aid

EN | Click for your language

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ES | Haga clic para su idioma

PT | Clique para o seu idioma

Care Guide.

Receiver-In-The-Canal (RIC) Hearing Aid

Content in kit

52 Cleaning wipes

1 Cleaning spray

1 Microfiber cloth

3 Boxes of CeruStop filters
(a total of 24 wax filters)

1 Brush


1 Vent brush




2 Green vent
cleaning wires

☐ Coupling: shell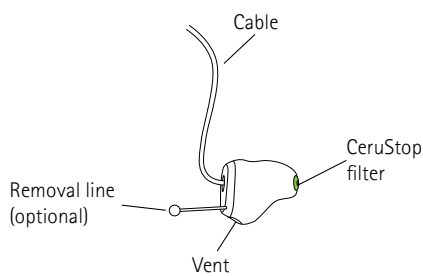

☐ Coupling: dome

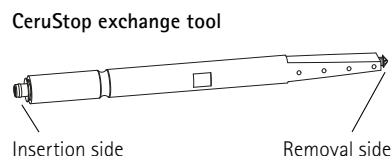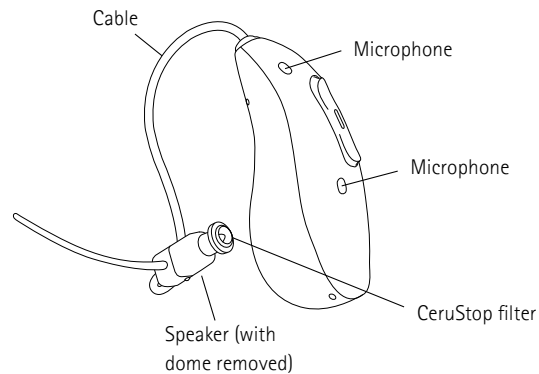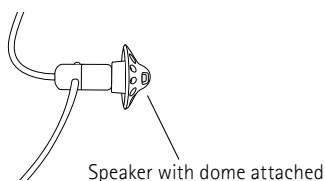Caring & cleaning

Congratulations on prioritizing your hearing health with your new hearing aids! While these devices are designed for long-lasting reliability and durability, it's essential to protect them through regular maintenance. Taking just a few minutes each day to care for your hearing aids will help ensure they continue to perform at their best.

Earwax is a natural secretion that helps protect your ears. However, too much earwax can cause issues for your hearing aids, like blocking sound or making it unclear. Most hearing aids have filters to keep wax out. It's important to check and change these filters regularly to ensure your hearing aids work well.

Your Care Guide offers you helpful steps for the proper maintenance of your hearing aids. Additionally, consider utilizing the included monthly calendar to assist in keeping track of your hearing aid maintenance.

Daily

Inspect your hearing aid before use daily to ensure it is clean. If needed, spray a tissue with the cleaning spray to wipe the shell and hearing aid body. Remove any earwax accumulated at the tip of the dome or shell. After, wipe the hearing aid using a clean and absorbent cloth or the included microfiber cloth.

Important: Never spray your hearing aids directly with the cleaning spray.


If you notice any debris gently brush the surface of the hearing aid or shell. If you have a dome avoid using a brush but rather wipe it off with a clean cloth.

Important: Be careful not to brush into the microphone openings to avoid damaging them.

Weekly

Use a cleaning wipe to remove any wax or debris from the hearing aids. After, wipe the hearing aids using a clean and absorbent cloth or the included microfiber cloth.

If you have a shell coupling use the vent cleaning wire or vent brush to clear the vent of any wax or debris by threading through the vent and wiping the wire or brush with a cleaning wipe.

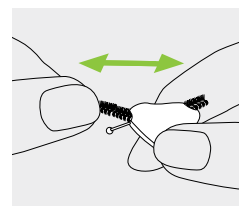Watch the instruction video on how to clean your hearing aid.

Monthly: Change of the Dome

If you have a dome coupling replace the domes every month to ensure optimal performance.

Important: When changing the dome, avoid pulling or applying excessive force to the speaker (including the cable).


Watch the instruction video on how to replace dome.



1

Hold the speaker in one hand and the dome in the other.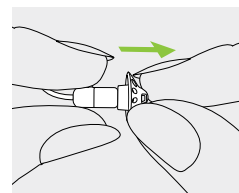


2

Gently pull off the dome to remove it.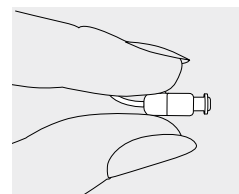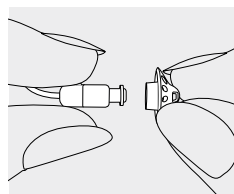



3

Pick up a new dome. Slide the dome over the speaker.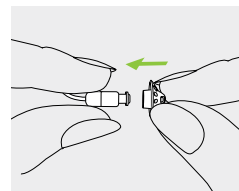

4

The speaker and the dome should fit perfectly together.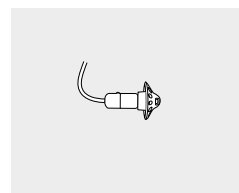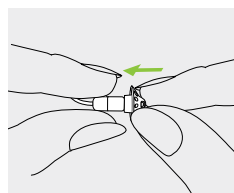

Monthly: Change of the CeruStop filter

Your CeruStop filter should be changed when:

- You observe that the filter positioned in the receiver is obstructed.
- The hearing aid does not produce normal sound.
- At least every month.


Watch the instruction video on how to change the wax filter.

1


First, if you have a dome, remove it by pinching the sides and pulling it off.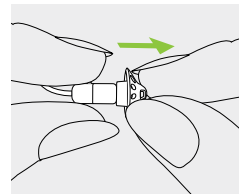


2

Insert the removal side of the exchange tool into the used wax filter.

Coupling: dome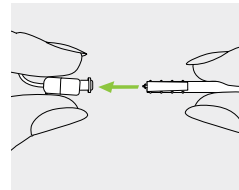


Coupling: shell

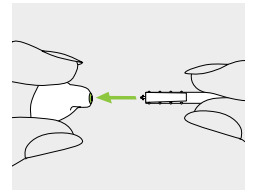
3

Carefully pull and remove the wax filter straight out of the hearing aid. Please do not twist the wax filter when removing.

Coupling: dome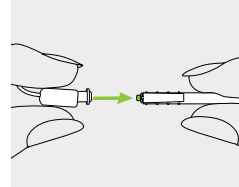


Coupling: shell

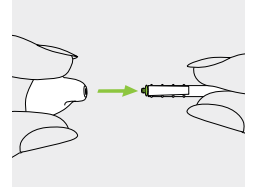
4

To insert the new wax filter, gently push the insertion side of the exchange tool straight into the hole.

Coupling: dome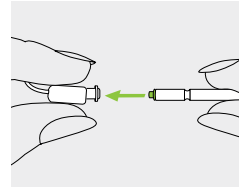


Coupling: shell

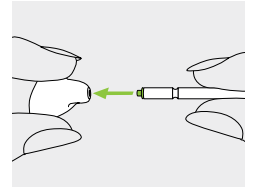
5

Pull the tool straight out. The new wax filter will remain in place. If you have a dome coupling reattach it now.

Coupling: dome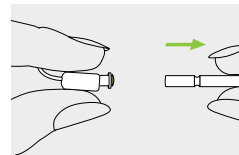

Coupling: shell

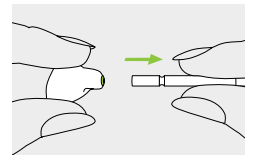Make sure you change your filter and dome (if applicable) at least monthly. Mark the calendar below when done.

JAN	FEB	MAR	APR	MAY	JUN
JUL	AUG	SEP	OCT	NOV	DEC

Looking for more support with your hearing aids?

Visit our support page at phonak.com/SupportHome

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