


# Phonak Audéo™ Infinio.

Quick Guide to get you started




# Getting started

Left & right hearing aid markings

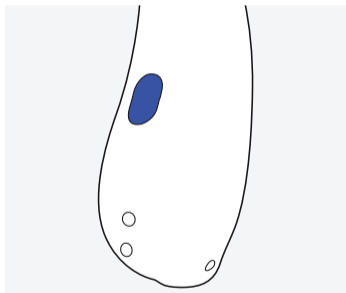

Blue marking for **left** hearing aid.

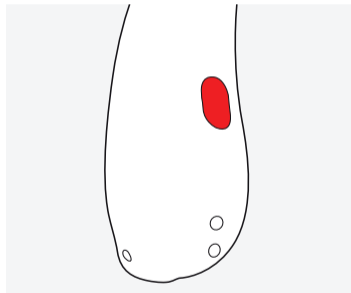

Red marking for **right** hearing aid.

# Charging your hearing aids

- ① Before using your hearing aid for the first time, it is recommended to charge it for 3 hours.

Solid green light = fully charged

Blinking light = charging




## Multi-function button

**On/Off:** Press lower part for 3 seconds until the indicator light blinks. On: solid green; Off: solid red

**Phone calls:** If paired with a Bluetooth® enabled phone, a short press to accept and a long press to reject a call.

**Enter flight mode:** When device is off, press the lower part of the button for 7 seconds until the solid orange light appears. Then release the button.

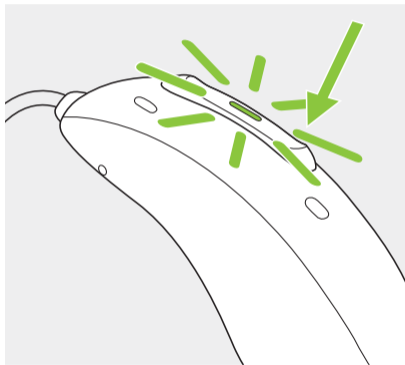

# Cleaning the hearing aid


## Daily

**Hearing aid:** Inspect the earpiece for earwax and moisture and clean the surfaces with a lint-free cloth. Never use cleaning agents such as household detergents, soap, etc. for cleaning your hearing aid. It is not recommended to rinse with water. Do not use a brush on the receiver. For intensive cleaning, please consult your hearing care professional.


**Charger:** Make sure that charger inserts are clean and clean the surfaces with a lint-free cloth.

## Exchanging ear wax protection


1. Insert the removal side of the exchange tool into the used earwax protector. The shaft of the holder should touch the rim of the earwax protector.

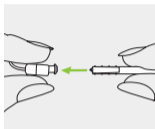


2. Carefully pull and remove the earwax protector straight out of the speaker. Please do not twist the earwax protector when removing.

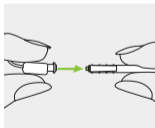

3. To insert the new earwax protector, gently push the insertion side of the exchange tool straight into the hole of the speaker until the outer ring is perfectly aligned.

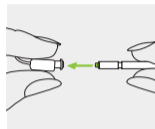

4. Pull the tool straight out. The new earwax protector will remain in place. Now attach your earpiece to the speaker.

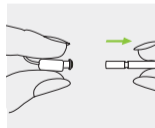

# Connecting to the myPhonak app to control your hearing aid(s)


1. Search for "myPhonak" on the Google Play™ Store or Apple App Store® or open your camera app to scan the QR code and then install the app.



**myPhonak**  
Sonova AG




2. Open the app and restart your hearing aids to enable pairing mode for 3 minutes. See step 2 in "Connecting to your phone for phone calls and audio" section for instructions.
3. Tap "Pair hearing aids" when you reach the "Home" screen and the app will guide you through the process of pairing and restarting devices.

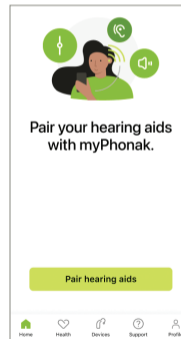

## Connecting to the myPhonak app to control your hearing aid(s), continued

4. Wait for your hearing aid(s) to show up. If you wear two hearing aids, make sure both display before continuing. If you only see one hearing aid, reboot both and try again. Once your device(s) are found, tap on them to pair to the app.

After pairing the app, you will see two new Phonak connections in your phone's Bluetooth menu. These connections are for your app to communicate to each device. They may say "Not Connected." That is typical as they will automatically connect when the app is opened. Do not remove them unless you are troubleshooting your app connection.

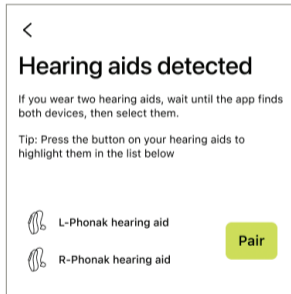

## Connecting to your phone for phone calls and audio

If you use the myPhonak app to pair your hearing aids with your phone, you can skip the instructions in this chapter. The pairing happens automatically through the app.

1. Open the Bluetooth® menu on your phone (often found in "Settings"). Make sure Bluetooth is on.

iPhone®



Android™





2. Turn your hearing aid(s) off and back on again. For rechargeable hearing aids, place them in the charger for at least 3 seconds and then remove them. This will enable pairing mode for 3 minutes and make them discoverable on your phone.



## Connecting to your phone for phone calls and audio, continued

3. One hearing aid will show in the devices list (typically as "R-Phonak hearing aid"), but your hearing care professional may have changed the name or side. If you do not see a device, tap "Scan" or "Search". When your device appears, tap on it to pair and then accept any Bluetooth pairing requests from the phone.

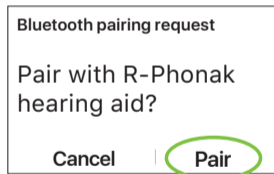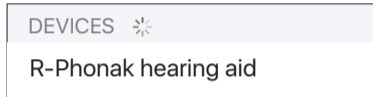

Note: only one pairing is required for phone calls and audio streaming. If you have two hearing aids, you will hear the audio in both hearing aids with the single pairing.

This quick guide is valid for:  
Phonak Audéo I-R  
Phonak Audéo I-Sphere

It is recommended to  
read the full user guide.



## Additional information



Visit [phonak.com/user-guides](https://phonak.com/user-guides) for the product user guide or scan the QR code.



Visit [phonak.com/support-bluetooth-setup](https://phonak.com/support-bluetooth-setup) to watch a video of this process or scan the QR code.



Visit [phonak.com/AppSetup](https://phonak.com/AppSetup) to watch a video of this process or scan the QR code.



Manufacturer:  
Sonova AG  
Laubisrütistrasse 28  
CH-8712 Stäfa  
Switzerland  
phonak.com



028-2708-03/V2.00/2025-07/hc  
© 2024-2025 Sonova AG. All rights reserved.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks by Sonova AG is under license.  
iPhone is a trademark of Apple Inc., registered in the U.S. and other countries and regions.  
App Store is a service mark of Apple Inc., registered in the U.S. and other countries and regions  
Android and Google Play are trademarks of Google LLC.

**sonova**  
HEAR THE WORLD

