







# myPilot Quick Guide

## Description

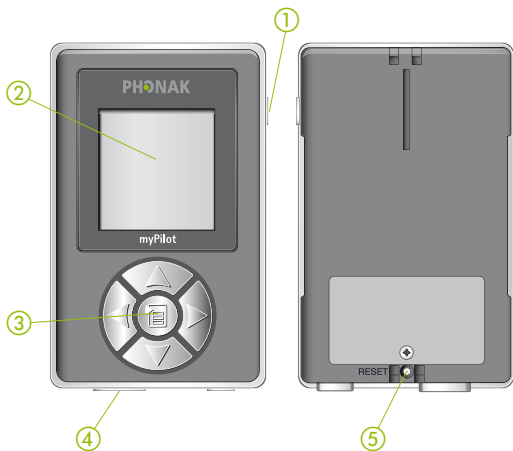

- ① ON/OFF button
- ② Color LCD display
- ③ Keypad

Up     Volume up  
Down    Volume down  
Right    Next program  
Left    Previous  
          program  
Center    Menu/select/home

- ④ Mini USB connection
- ⑤ Reset button

PHONAK

life is on

---

## Getting started

### Charging the battery

When charging myPilot for the first time, leave it to charge for five hours, even if the battery symbol stops flashing.




### Turning myPilot ON or OFF

To turn myPilot ON or OFF, press the ON/OFF button for two seconds until you see the display turn ON or OFF.




### “Home” button

From any screen or menu level, press the middle button  for 4 seconds to always return to the default settings of your hearing system and the Home screen of myPilot.

---

## Using myPilot


### Selecting your preferred Home screen


To select different Home screens, enter the menu by pressing the Center button  then select *Settings*, then *Home screen*.

The "standard"-mode gives you the most complete overview of volume and program, including iconic pictures of the programs.



In the "direct control"-mode each button directly selects the indicated program number. This mode is for the program and volume control only with no menu or ZoomControl access.

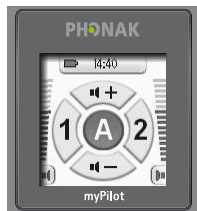

 The screens of your myPilot may look different, depending on your preferences set by your hearing care professional.



---

## Using myPilot

### "Standard"-mode

The "standard"-mode gives the full overview including program and volume information.



- Press the left or right arrow (◀/▶) to scroll through the hearing programs
- Press the up-/down arrow (△/▽) to change the volume. See page 6 for details
- Short press middle button  to access the sub-menus
- Press middle button  for 2 seconds to change to the ZoomControl screen. See page 7 for details




---

## Using myPilot







### “Direct control”-mode

In the “direct control”-mode each key selects the indicated program number.

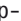

Programs	Left 	Middle 	Right 
Short press	1	A	2
2 second press	3	4	5

The program names are shown for 10 seconds after a new program is selected.

- Press the up-/down arrow (/) to change the volume.
  - Press the left and right button (/) simultaneously for 2 seconds to skip the direct control-mode
-  The available hearing programs are set by your hearing care professional.
-  In the direct control-mode, the screens for volume control, sub-menus and zoom control are not available. Quick access however is available.

## Changing the volume

In the "standard"-mode Home screen go to the Volume screen by pressing either  $\triangle$  or  $\nabla$ . Once in the Volume screen, you can change the volume of both hearing instruments simultaneously by pressing  $\triangle$  to increase and  $\nabla$  to decrease the volume.



In a similar way you can change the volume settings of your left (or right) hearing instrument individually. Press  $\triangleleft$  to choose the left side (or  $\triangleright$  to choose the right side) after entering the Volume screen as described above. Then press  $\triangle$  to increase and  $\nabla$  to decrease the volume.








---

## Using myPilot

### ZoomControl (optional)

This special feature allows you to focus your hearing system to front, back, left or right.



- In the "standard" Home screen press the middle button  for 2 seconds to enter the ZoomControl screen
- Short press the left, up, right or down button (, ,  or ) to select left, front, right or back hearing direction
- Short press middle button  to access the sub-menus
- Press the middle button  for 2 seconds again to return to the program selection screen and to activate the last used program
- Press either the up or down button for 2 seconds to enter the volume control screen

**Note:** Ask your hearing care professional whether ZoomControl is available on your hearing instruments.

---

## Using myPilot

### Special functions

From the menu *Settings* → *Quick access* → *ON/OFF button* you can select a special function associated with a short press of the ON/OFF button.

#### Quick access

Short press the ON/OFF button to activate the stored program/volume settings even from sleep mode.

#### Status readout

Short press the ON/OFF button to start the hearing instrument status readout process.


### Visit card

If set, find your hearing care professional contact in the main menu under *Contact*.



---

## Important information

For your safety and to benefit from all the features of your myPilot, please read the myPilot User Guide carefully. Pay particular attention to the "Important information" section and the following symbol  which refers to associated warnings.

Please ask your hearing care professional for more help.

[www.phonak.com](http://www.phonak.com)

