


4

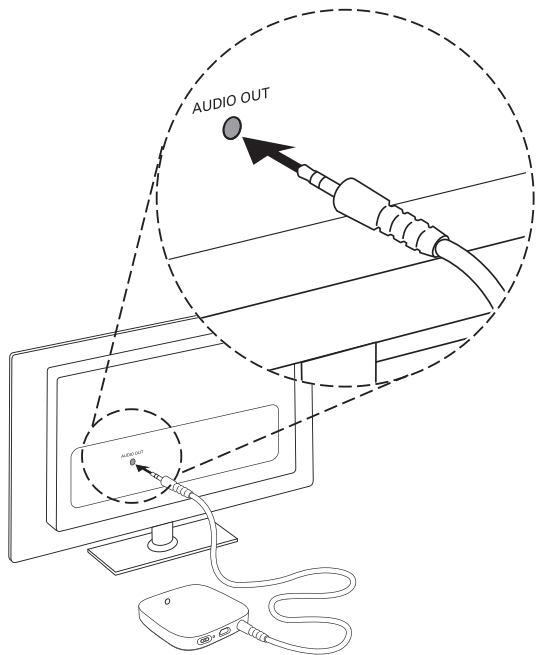


**Manufacturer:**  
Sonova AG  
Laubisrütistrasse 28  
CH-8712 Stäfa  
Switzerland

029-0518/V1.00/2017-06/ma © Sonova AG All rights reserved

# TV Connector

How to setup/Installationsanleitung  
Guide d'installation/Guía de instalación

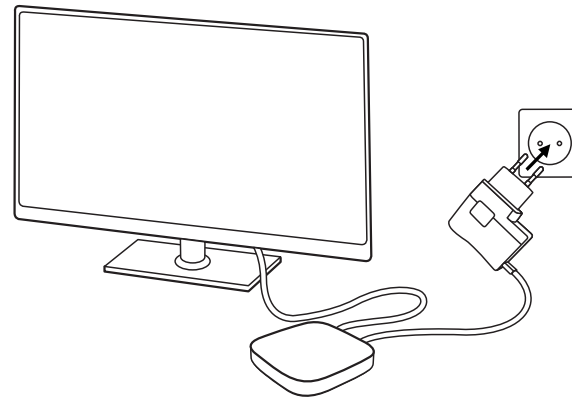

VIDEO

[www.phonak.com/TVConnector-install](http://www.phonak.com/TVConnector-install)




1

Insert USB cable (bigger plug) into power supply

USB Kabel (grosser Stecker) an Netzteil anschliessen

Branchez le câble USB (fiche la plus grande) dans l'adaptateur secteur

Conectar el cable USB (conector grande) de la fuente de alimentación

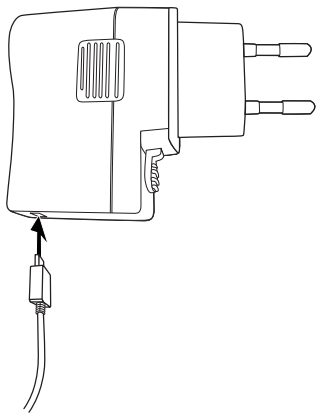


2

Plug power supply into mains.

Netzteil an Steckdose anschliessen.

Branchez l'adaptateur secteur dans une prise électrique

Enchufe la fuente de alimentación a la toma de corriente

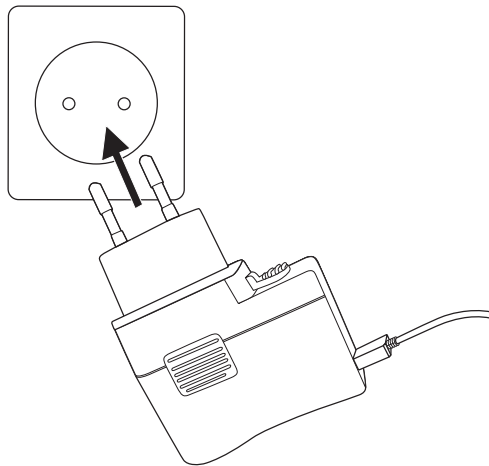

3

Insert USB cable (small plug) into TV Connector

USB Kabel (kleiner Stecker) an TV Connector anschliessen

Branchez le câble USB (fiche la plus petite) au TV Connector

Conecte el cable USB (conector pequeño) de la TV Connector

