

# Creating a Roger™ network in the classroom


Reference Sheet

roger



All Roger for Education devices were designed to be easy to use with one another. All that is required is to connect them in a network to work with each other. Depending on what devices you have, there are multiple ways to connect them. The choice is yours.

## Add Roger devices via Connect

Hold the Roger educational microphone within 4" (10 cm) of the following Roger devices to connect them to the classroom network, and click **Connect** or tap the **Connect** icon .

- Roger ear-level receivers
- Roger MyLink
- Roger Focus
- Roger educational microphones
  - Roger Touchscreen Mic
  - Roger Pass-around
  - Roger inspiro\*
  - Roger DynaMic\*
- Roger Multimedia Hub
- Roger AudioHub\*
- Roger DigiMaster 5000\*\*, 7000\*\* loudspeakers
- Roger DigiMaster X

\* When connecting these devices into a MultiTalker Network with a Touchscreen Mic as Primary, follow the steps on the Touchscreen Mic display after tapping **Connect**.

\*\* When connecting a Roger DigiMaster loudspeaker, position the Roger microphone near the top of the Roger DigiMaster.

## Add Roger devices via Roger WallPilot



**Roger receivers** – The children simply need to walk up to the Roger WallPilot (ears and receivers within 12" (30 cm) of WallPilot) and face it for a couple of seconds.

**Roger microphones and media transmitter** – The following Roger microphones and multimedia transmitter can be connected by simply turning them off and then back on within 4" (10 cm) of the Roger WallPilot:

- Roger Touchscreen Mic
- Roger Pass-around
- Roger Multimedia Hub
- Roger inspiro
- Roger DynaMic