

Using advanced Roger™ Touchscreen Mic network options

Reference Sheet

roger



When multiple students and teachers are using Roger devices in one classroom, it may be necessary to do more than just Connect and go. Sometimes the teacher may want to check multiple devices each morning or a co-teacher may want to talk to only one or a portion of Roger students. With the Roger Touchscreen Mic it is easy to do with a few taps.



Disconnect allows a teacher to disconnect a device from the network. If a child no longer needs to listen to the Roger signal, for instance when a child would be working independently, the teacher can still teach other Roger listeners.

- Swipe to second screen and tap the **Disconnect** icon within 4" (10 cm) of the Roger device to be disconnected.
- Reconnect the child to the network either by tapping **Connect** within the same distance or having them connect via the Roger WallPilot.



SubNet is used for a teacher to talk to one or a few of the students, without the other children listening for a short period of time. This is great for pull-out work and teacher's aids. The SubNet feature also allows the secondary mic to be used in Small Group mode for a portion of students.

- Tap the **SubNet** icon on the home screen.
- **Connect** receivers of the students to the microphone by tapping Connect icon within 4" (10 cm) of receivers.
- Wear the microphone on the lanyard for teacher instruction, or place it on the table to access Small Group mode for group activities.
- When finished, tap **Network** icon and reconnect the children's receivers to the network either by tapping **Connect** or having them connect via the Roger WallPilot.



JoinNet is used to add a Roger Touchscreen Mic to a desired network without disturbing the teacher during the lecture. This is an alternative if no Roger WallPilot is available.

- Tap **JoinNet** icon within 4"(10 cm) of any Roger device in that network.



NewNet is used when a Roger Touchscreen Mic needs to leave one network and/or reset the classroom network.

- Tap **NewNet** icon
- Connect any Roger devices to this new network by tapping **Connect** within 4" (10 cm) of the Roger Touchscreen Mic



Mass Connect is used to connect multiple devices in close proximity very quickly. This option must be configured in the Settings menu.

- Arrange all Roger receivers near each other and make sure they are powered on.
- Tap the **Mass Connect** and sweep within 4" (10 cm) over all of the devices. The devices should connect one after the other as the Roger Touchscreen Mic goes by.
- Tap on the pop-up screen to stop **Mass Connect**.
- For confirmation of connection, tap List devices.
- If a device did not connect, tap the **Mass Connect** icon near it again.



List devices: A quick way to visualize which Roger devices are in close range, and which are connected to the Roger Touchscreen Mic.

- All Roger devices shall be within 4" (10 cm) to the Roger Touchscreen Mic
- Tap **List devices**
- All nearby Roger devices will be listed (the time may vary depending on number of devices to find). An available device will appear in the list greyed-out and a connected device will be listed in black with a connect symbol.
- If desired Roger device is not connected, tap the **Connect** icon within 4" (10 cm) of device.