

# Using Roger inspiro and Roger design-integrated receivers

## Reference Sheet



First name: \_\_\_\_\_

Last name: \_\_\_\_\_

School: \_\_\_\_\_

Teacher: \_\_\_\_\_


Roger contact: \_\_\_\_\_

Phone: \_\_\_\_\_

Date: \_\_\_\_\_

## Microphone: Roger inspiro

### Instructions for use

1. To switch on, push the slider to the left until the green mark appears and the screen is illuminated.
2. Clip microphone within 7 inches (20 cm) of mouth or hang boom mic appropriately on ear, with microphone in front of mouth.
3. Does the student need to be connected? If so, position the transmitter within 4 inches (10 cm) of one Roger X receiver. Press the right **Connect** softkey. Repeat on other side if the student has 2 Roger receivers.
4. Lock buttons by pressing the , then press **OK**. A key symbol will appear in the status bar.
5. At the end of every day, switch the inspiro off by pushing the top slider to the right, and connect the Mini-USB plug from the charging unit to the Roger inspiro.

Serial number: \_\_\_\_\_



## Receivers: Roger design-integrated

### Instructions for use

1. Design-integrated Roger receivers should be placed according to their User Guides.
2. Removal should not be necessary at school. (These receivers were designed for permanent placement on hearing aids).

Serial numbers

Right: \_\_\_\_\_ Left: \_\_\_\_\_

