

Creating a Roger MultiTalker Network

Reference Sheet



Multiple microphones can be used in a Roger Network, allowing multiple teachers, children, and audio devices to broadcast to Roger listeners. Below you will see there are multiple ways to create or join a MultiTalker Network (MTN). MTN connections are always retained, even when microphones are turned off.

Add Roger microphones/AudioHub/DigiMaster via Connect

1. Position the primary Roger inspiro within 4 inches (10 cm) of any Roger microphone or Roger AudioHub and press **Connect**.
Roger inspiro will confirm the connection via the LCD screen.
2. Repeat with any other Roger devices.



Add Roger microphones via Roger WallPilot

1. Position the Roger microphone within 4 inches (10 cm) of any Roger WallPilot.
2. Turn microphone off then on again.
3. The microphone is now connected to the classroom network.



Add Roger inspiro via JoinNet

1. Position Roger inspiro within 4 inches (10 cm) of any Roger device.
2. Press the **JoinNet** softkey.
3. The Roger inspiro is now connected to the classroom network.

