

# Using Roger inspiro and Roger X

## Reference Sheet



First name: \_\_\_\_\_

Last name: \_\_\_\_\_

School: \_\_\_\_\_

Teacher: \_\_\_\_\_


Roger contact: \_\_\_\_\_

Phone: \_\_\_\_\_

Date: \_\_\_\_\_

## Microphone: Roger inspiro

### Instructions for use

1. To switch on, push the slider to the left until the green mark appears and the screen is illuminated.
2. Clip microphone within 7 inches (20 cm) of the mouth.  
Or hang boom mic appropriately on ear, with microphone in front of mouth.
3. Does the student need to be connected? If so, position the transmitter within 4 inches (10 cm) of one Roger X receiver. Press the right **Connect** softkey. Repeat on other side if binaural.
4. Lock Roger inspiro's buttons by pressing the , then press **OK**. A key symbol will appear in the status bar.
5. At the end of every day, switch Roger inspiro off by pushing the top slider to the right, and connect the Mini-USB plug from the charging unit into the Roger inspiro.

Serial number: \_\_\_\_\_



## Receiver: Roger X

### Instructions for use

1. Place Roger X on hearing aid using appropriate audioshoe. It is now ready for use.
2. If the receiver must remain at school, remove audioshoe and Roger X from the hearing instrument at the end of the school day.

Serial numbers

Right: \_\_\_\_\_ Left: \_\_\_\_\_

