

Impression Measurement Card

Below are instructions to help you use your patient's ear impression to determine if their canal size will accommodate the build for either a custom product or the receiver-in-canal build.

The blue marks aid in determining length and the cut-outs indicate width. Receiver sizes shown include those for SDS 4/5/6 and are S, M, P.


Checking the length:

Hold the impression next to the blue marking on the right side of the card. Measure from the first bend of the ear canal. If the ear impression is longer than the blue marking, the ear canal is long enough to hold the receiver size.


Checking the width:

1. Find the narrowest part of the ear impression
2. Place the width marker over this
3. If the width marker doesn't slide over this easily, the canal should be wide enough for the receiver size you are after.
4. If the width marker slides over this easily (i.e. the width is wider than the ear canal), the receiver is likely too large for the ear canal.


Examples

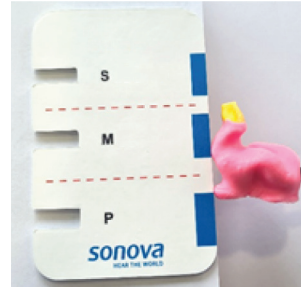


Outcome =
Length OK for M receiver

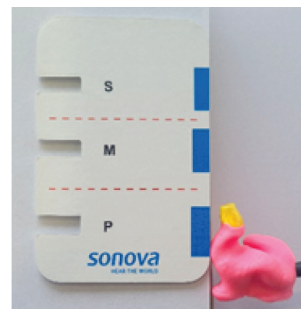


Outcome =
Length is OK for P receiver

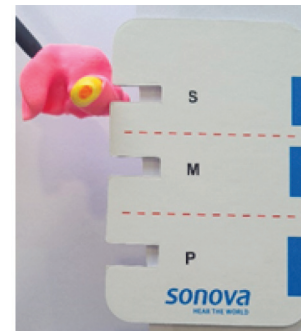

Outcome =
Width OK for S receiver

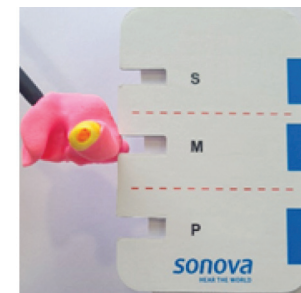

Outcome =
Width OK for M receiver

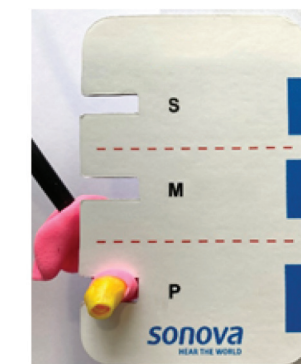

Outcome =
Canal too narrow for
P receiver