

Roger NeckLoop for speech-to-text

Installation instructions

Roger NeckLoop can be connected to a computer, tablet or smartphone to generate live, automated captions using 3rd party speech-to-text software.


Preparations

1. Ensure your Roger NeckLoop and your Roger microphone are charged.
2. Ensure you have a 3rd party speech-to-text, live transcribe or dictation application installed on your computer, smartphone or tablet.

Connection to computer


You need: The included USB-C to USB-A cable.

1. Connect the USB cable to the Roger NeckLoop and your computer:

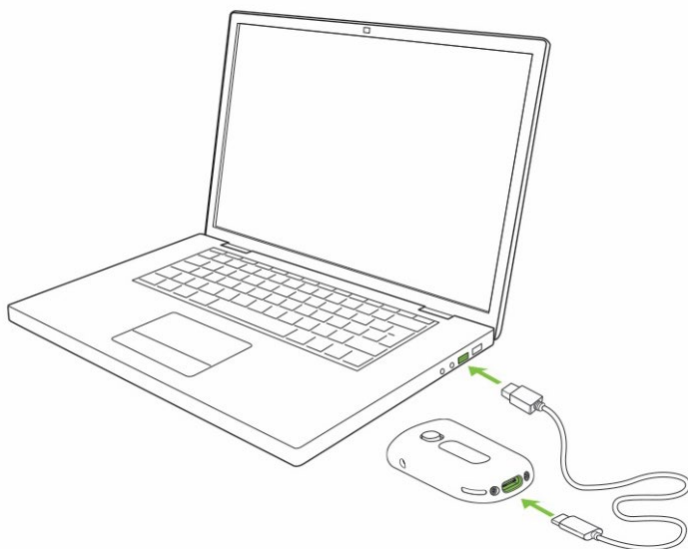

2. Turn Roger NeckLoop on.

3. On your computer, check that Roger NeckLoop is selected as the input source (sound settings):

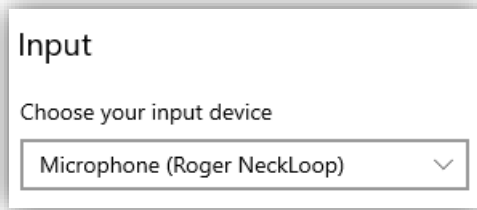

4. Run the speech-to-text software (3rd party application) on your computer. Alternatively, you can also run the 'Dictate' feature in Microsoft Word:




5. Now speak into the Roger microphone and the speech-to-text software or MS Word will transcribe the spoken words into text.

Connection to smartphone or tablet with a USB-C socket

You need: The included USB-C to USB-A cable and an *On-the-Go (OTG) adapter*, OR a *USB-C to USB-C cable*.

1. Connect Roger NeckLoop to your smartphone or tablet with an OTG-adapter and the included USB cable, OR use a USB-C to USB-C cable.
2. Turn Roger NeckLoop on.
3. Run the speech-to-text app (e.g. Google Transcribe) on your smartphone or tablet.
4. If necessary, select Roger NeckLoop as input source.
5. Speak into the Roger microphone and the speech-to-text app will transcribe the spoken words into text.

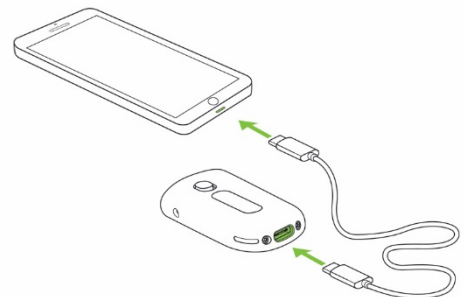

Connection to smartphone or tablet with a Lightning socket

You need: The included USB-C to USB-A cable and a *Lightning to USB 3 Camera Adapter*. Your phone charger.

1. Connect the adapter to your smartphone.
2. Connect your phone charger to the adapter.
3. Connect Roger NeckLoop to the adapter with the included USB-C to USB-A cable.
4. Turn Roger NeckLoop on.
5. Run the speech-to-text app on your smartphone or tablet.
6. If necessary, select Roger NeckLoop as input source.
7. Speak into the Roger microphone and the speech-to-text app will transcribe the spoken words into text.

