

Basic ZoomLink+ Operation Instructions



ZoomLink+ On / Off

- Press and hold one of the three microphone buttons for two seconds
- The transmitter should be switched on when the student is involved in the communication and switched off when they are not.



Wearing ZoomLink+

- Worn on breastbone as shown in picture
- Buttons facing out as shown, LCD screen towards body
- Distance from ZoomLink to speakers mouth should be 20 to 30cm
- Recommended to operate in SuperZoom mode




Holding ZoomLink+

- Hold in hand as shown
- Do not touch the microphones shown in green. This will create an unpleasant sound for the student



Charging ZoomLink+

- Plug in backpack as shown (1)
- Plug in USB charger
- Plug into wall

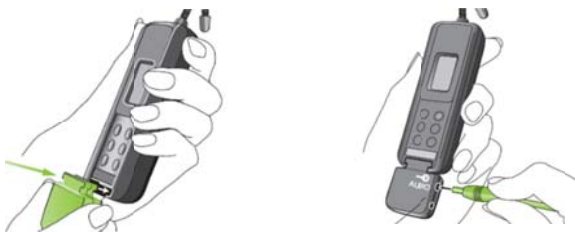

Using the Audio Input

- Plug in backpack as shown
- Plug audio cable to backpack as shown and plug the other
- The cable has an optional RCA adapter to plug into audio connections



Super Zoom Microphone Mode

Best for

- When worn around neck
- When pointing at what you want to hear
- Noisy situations



Zoom Microphone Mode

Best for

- Group work in noise (must be pointed in the direction you want to hear)
- Noisy situations



Omni Microphone Mode

Best for

- Small group work in quiet
- All quiet situations