











Phonak Wireless Accessories for Venture hearing aids

		DECT II	EasyCall II	ComPilot II	ComPilot Air II	PilotOne II	RemoteControl App	TVLink II basestation	TV and Music Set	Conversation Set	RemoteMic
Use case											
Phone calls	Home phone	•									
	One cell phone		•	•	•			•	•		
	Multiple cell phones			•	•			•	•		
	VoIP (Skype) / Office phone			•③	•③			•③	•③		
Conversation over distance	Calm environment								•	•②	
	In some noise								○④	○④	
Streaming	TV/Music up to 3h per day			•	•①			•②	•	•①	
	TV/Music more than 3h per day			•①				•②	•		
Remote control				•	•	•	•②	•	•		

Additional information

Connectivity / Supported inputs	Bluetooth		•	•	•				•	•	
	Europlug (Roger/FM)			•					•		
	3.5 mm audio plug			•				•	•		
	Digital optical/coaxial							•	•		

Phone solution guide		Streaming guide		Conversation over distance guide	
Phonak DECT II	Use as home and small office solution for fixed installation.	ComPilot II	Use for long streaming time per day when watching movies and listening to music.	Phonak RemoteMic	Use for listening to a speaker over distance in calm situations.
Phonak EasyCall II	Use when always using the same cell phone.				
Phonak ComPilot II or ComPilot Air II	a) When using 2 cell phones in parallel or if you switch between cell phones. b) When doing hands-free calls e.g., while driving a car.	ComPilot Air II	Use for up to 4h streaming time per day for occasional use of watching TV or mainly doing phone calls.	Phonak Roger devices	Use for listening and conversation over distance in noisy environments.














① requires a TVLink II for TV watching

② requires a Phonak streamer (ComPilot II or ComPilot Air II)

③ requires a USB adapter 076-0856 / phone adapter 076-0031

④ for conversation over distance in loud noise we recommend Roger products

Compatibility overview Wireless Accessories

		Hearing aid generation								
		Phonak Venture Compatible					Phonak Quest/Spice compatible			
		PilotOne II	ComPilot II	ComPilot Air II	DECT II	EasyCall II	EasyCall	PilotOne	ComPilot	DECT CP1
										
Extended streamer solutions	TVLink S 		•	Streaming ok, charging slot does not fit					•	
	TVLink II 		•	•					•	
	RemoteMic 		•	•					•	
	RemoteControl App 	②	•	•						

① A choice of phone hard cases for EasyCall can be found on <http://www.phonak.com/easycall>

② The RemoteControl App runs on Apple iOS7 or Android 4.0 and higher.