


5

Usage  
Benützung  
Utilisation  
Utilización

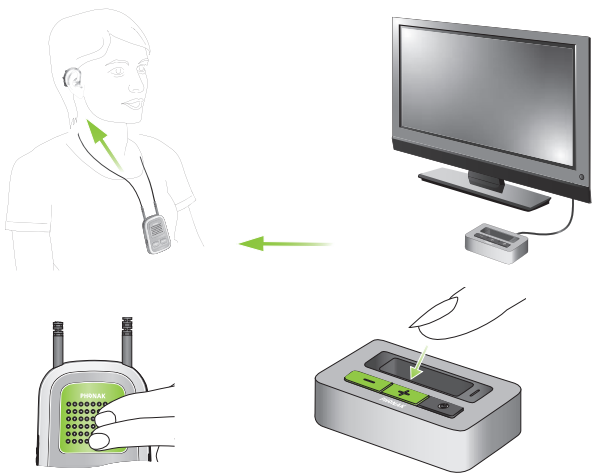


Pause / Resume  
Pause / Abspielen  
Pause / Diffuser  
Pausar/ Reanudar

Volume setting  
Lautstärke einstellen  
Réglage de volume  
Ajuste de volumen



6

Charging ComPilot  
ComPilot Aufladen  
Chargez ComPilot  
Cargar el ComPilot

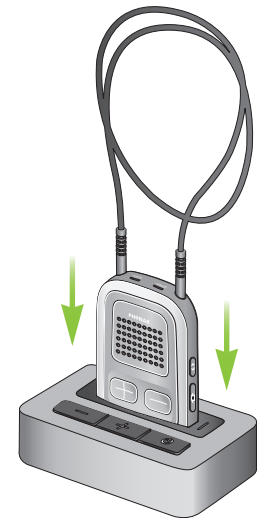

End

The Phonak TVLink S user guide will give you detailed information about the installation.

Die Phonak TVLink S Gebrauchsanweisung informiert detailliert über die Installation.

Le mode d'emploi Phonak TVLink S vous donne des informations détaillées sur l'installation.

Las instrucciones de uso del Phonak TVLink S le darán una información más detallada sobre su instalación.

### LED status

- ON  
Ein  
Marche  
Encendido
- No audio signal  
Kein Audiosignal  
Pas de signal audio  
No hay señal audio
- Audio transmission  
Tonübertragung  
Transmission audio  
Transmisión audio
- Pause  
Pause  
Pause  
No transmisión
- Pairing  
Koppeln  
Couplage  
Emparejamiento

Manufacturer:  
Phonak AG  
Laubisrütistrasse 28  
CH-8712 Stäfa  
Switzerland

EU Representative:  
Phonak GmbH  
D-70736 Fellbach-Oeffingen  
Germany

www.phonak.com 0678

029-1069/V1-00/2011-09/A+/W/FO Printed in Switzerland  
©Phonak AG All rights reserved

## Phonak TVLink S

Quick setup guide / Schnellstartanleitung  
Guide d'installation rapide / Guía rápida de conexión

60 sec  
plug & play



PHONAK life is on


Start

Components  
Komponenten  
Composants  
Componentes





1

Connect power and audio cable  
Netzteil und Audio-Kabel anschliessen  
Connectez le câble d'alimentation et le câble audio  
Conecte el cable de conexión y el de audio

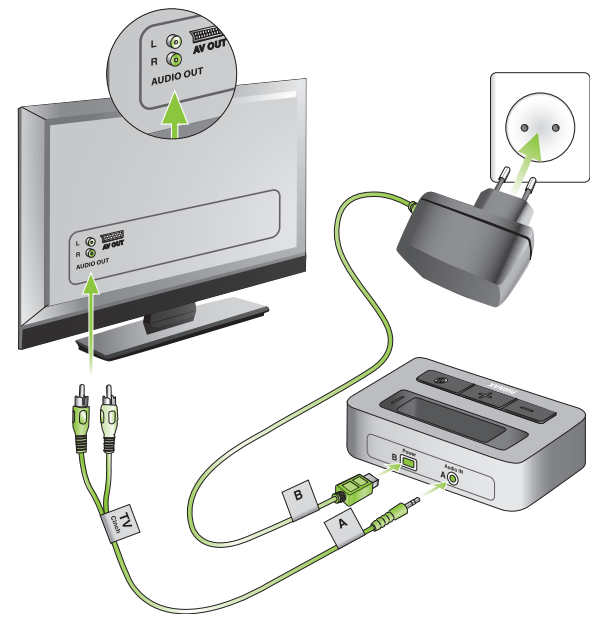

2


Switch ON both devices  
Beide Geräte einschalten (ON)  
Mettez les deux appareils en marche (ON)  
Encienda los dos dispositivos

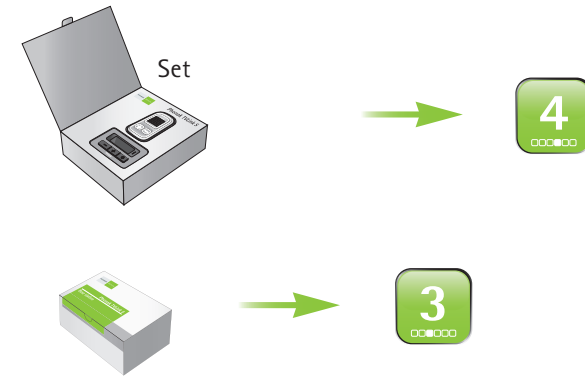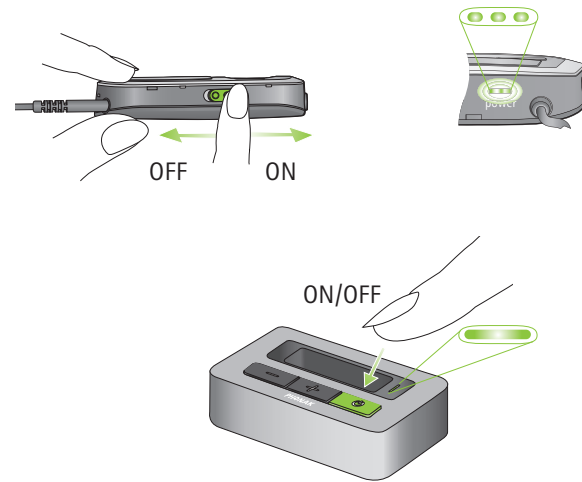

If you obtained TVLink S in the complete set go to 4.  
Direkt zu 4. wenn TVLink S im Set gekauft wurde.  
Si vous avez acheté le kit TVLink S complet allez directement au point 4.  
Si usted ha obtenido TVLink S en el set completo vaya al paso nº 4.

3

Pairing  
Kopplung  
Couplage  
Emparejamiento

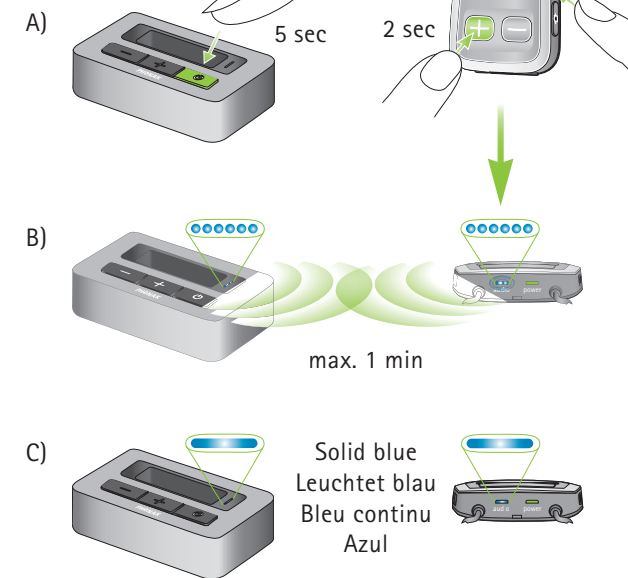

4

Wearing ComPilot  
ComPilot tragen  
Portez ComPilot  
Llevar ComPilot

